

## Available ribbon commands

See also the [tab inventory page](#).

| Ribbon section | Command name                   | Parameters         |
|----------------|--------------------------------|--------------------|
| File menu      | custom-panel-algo-ichi         | props=             |
|                | custom-panel-algo-ni           | props=             |
|                | custom-panel-ai-prov-mindpeak  | props=             |
|                | custom-panel-ai-prov-spintellx | props=             |
|                | file                           |                    |
|                | about-us                       |                    |
|                | show-eula                      |                    |
|                | manual                         |                    |
|                | sign-out                       |                    |
| Ribbon section | Command name                   | Parameters         |
| Home tab       | simple-layout                  |                    |
|                | detailed-layout                |                    |
|                | detailed-layout                |                    |
|                | viewport-toggle-overview       |                    |
|                | viewport-toggle-slide-path     |                    |
|                | viewport-toggle-barcode        |                    |
|                | viewport-toggle-rotation-flip  |                    |
|                | add-tab                        |                    |
|                | reset-layout                   |                    |
|                | viewport-toggle-zoom           |                    |
|                | viewport-toggle-scalebar       |                    |
|                | viewport-toggle-label          |                    |
|                | panel-toggle-color             |                    |
|                | panel-toggle-search            |                    |
|                | open-folder-in-grid            |                    |
|                | prev-directory                 |                    |
|                | next-directory                 |                    |
|                | prev-image                     |                    |
|                | next-image                     |                    |
|                | add-folder-bookmark            |                    |
|                | reset-grid                     |                    |
|                | clear-cell                     |                    |
|                | open-in-tab                    |                    |
|                | popout-info                    |                    |
|                | snapshot                       |                    |
|                | quick-snapshot                 |                    |
|                | toggle-toolbar                 |                    |
|                | increase-zoom                  |                    |
|                | decrease-zoom                  |                    |
|                | custom-zoom-1x                 | pma-magnification= |
|                | custom-zoom-2x                 | pma-magnification= |

| Ribbon section                  | Command name                                | Parameters         |
|---------------------------------|---------------------------------------------|--------------------|
|                                 | <a href="#">custom-zoom-5x</a>              | pma-magnification= |
|                                 | <a href="#">custom-zoom-10x</a>             | pma-magnification= |
|                                 | <a href="#">custom-zoom-20x</a>             | pma-magnification= |
|                                 | <a href="#">custom-zoom-40x</a>             | pma-magnification= |
|                                 | <a href="#">panel-toggle-color</a>          |                    |
|                                 | <a href="#">send-slide-to-tray</a>          |                    |
|                                 | <a href="#">send-to-tray</a>                |                    |
|                                 | <a href="#">send-dir-to-tray</a>            |                    |
|                                 | <a href="#">panel-toggle-tray</a>           |                    |
|                                 | <a href="#">share-slide-tray</a>            |                    |
|                                 | <a href="#">quick-share-folder</a>          |                    |
|                                 | <a href="#">quick-share-slide</a>           |                    |
|                                 | <a href="#">quick-share-slide-roi</a>       |                    |
|                                 | <a href="#">quick-share-grid</a>            |                    |
|                                 | <a href="#">quick-share-grid-rois</a>       |                    |
|                                 | <a href="#">quick-share-thumbnail</a>       |                    |
|                                 | <a href="#">share-slide</a>                 |                    |
| Ribbon section                  | Command name                                | Parameters         |
| <a href="#">Grid tab</a>        | <a href="#">toggle-1x1-grid</a>             |                    |
|                                 | <a href="#">toggle-2x2-grid</a>             |                    |
|                                 | <a href="#">toggle-3x3-grid</a>             |                    |
|                                 | <a href="#">sync-views-toggle</a>           |                    |
|                                 | <a href="#">reset-grid</a>                  |                    |
|                                 | <a href="#">clear-cell</a>                  |                    |
|                                 | <a href="#">open-in-tab</a>                 |                    |
|                                 | <a href="#">viewport-add-row</a>            |                    |
|                                 | <a href="#">viewport-remove-row</a>         |                    |
|                                 | <a href="#">viewport-remove-row</a>         |                    |
|                                 | <a href="#">viewport-remove-column</a>      |                    |
|                                 | <a href="#">flip-rotate-menu</a>            |                    |
|                                 | <a href="#">reset-flip</a>                  |                    |
|                                 | <a href="#">flip-horizontally</a>           |                    |
|                                 | <a href="#">flip-vertically</a>             |                    |
|                                 | <a href="#">reset-rotation</a>              |                    |
|                                 | <a href="#">rotate-clockwise-90</a>         |                    |
|                                 | <a href="#">rotate-counter-clockwise-90</a> |                    |
|                                 | <a href="#">rotate-180</a>                  |                    |
|                                 | <a href="#">zoom-in</a>                     |                    |
|                                 | <a href="#">panel-toggle-annotations</a>    |                    |
|                                 | <a href="#">save-annotations</a>            |                    |
|                                 | <a href="#">toggle-annotations</a>          |                    |
|                                 | <a href="#">toggle-annotation-text</a>      |                    |
|                                 | <a href="#">toggle-fingerprint</a>          |                    |
| Ribbon section                  | Command name                                | Parameters         |
| <a href="#">Annotations tab</a> | <a href="#">Cursor</a>                      |                    |

| Ribbon section                   | Command name                                       | Parameters                               |
|----------------------------------|----------------------------------------------------|------------------------------------------|
|                                  | <a href="#">delete-annotation</a>                  |                                          |
|                                  | <a href="#">delete-all-annotations</a>             |                                          |
|                                  | <a href="#">edit-annotation-text</a>               |                                          |
|                                  | <a href="#">merge-selected-annotations</a>         |                                          |
|                                  | <a href="#">union-selected-annotations</a>         |                                          |
|                                  | <a href="#">modify-selected-annotation</a>         |                                          |
|                                  | <a href="#">Point</a>                              |                                          |
|                                  | <a href="#">Circle</a>                             |                                          |
|                                  | <a href="#">Line</a>                               |                                          |
|                                  | <a href="#">Arrow</a>                              |                                          |
|                                  | <a href="#">Ellipse</a>                            |                                          |
|                                  | <a href="#">Rectangle</a>                          |                                          |
|                                  | <a href="#">Landmark1</a>                          |                                          |
|                                  | <a href="#">MultiPoint</a>                         |                                          |
|                                  | <a href="#">Polygon</a>                            |                                          |
|                                  | <a href="#">ClosedFreehand</a>                     |                                          |
|                                  | <a href="#">CompoundFreehand</a>                   |                                          |
|                                  | <a href="#">Freehand</a>                           |                                          |
|                                  | <a href="#">Wand</a>                               |                                          |
|                                  | <a href="#">Brush</a>                              |                                          |
|                                  | <a href="#">export-to-wkt</a>                      |                                          |
|                                  | <a href="#">export-to-mld</a>                      |                                          |
|                                  | <a href="#">export-to-annotations</a>              |                                          |
|                                  | <a href="#">export-to-aperio</a>                   |                                          |
|                                  | <a href="#">export-to-csv</a>                      |                                          |
|                                  | <a href="#">import-to-wkt</a>                      |                                          |
|                                  | <a href="#">import-to-mld</a>                      |                                          |
|                                  | <a href="#">import-to-halo</a>                     |                                          |
|                                  | <a href="#">import-to-csv</a>                      |                                          |
| Ribbon section                   | Command name                                       | Parameters                               |
| <a href="#">Pre-set tab</a>      | <a href="#">preset-Circle0237mm2</a>               | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Circle1mm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Circle2mm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Circle1cm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Square5mm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Square1cm</a>                   | pma-shape, pma-size-a, pma-size-b        |
|                                  | <a href="#">preset-Tumor</a>                       | pma-classification, pma-shape, pma-color |
|                                  | <a href="#">preset-Macro</a>                       | pma-classification, pma-shape, pma-color |
| Ribbon section                   | Command name                                       | Parameters                               |
| <a href="#">Conferencing tab</a> | <a href="#">create-conference</a>                  |                                          |
|                                  | <a href="#">join-conference</a>                    |                                          |
|                                  | <a href="#">start-conference</a>                   |                                          |
|                                  | <a href="#">pause-conference</a>                   |                                          |
|                                  | <a href="#">stop-conference</a>                    |                                          |
|                                  | <a href="#">toggle-everyone-control-conference</a> |                                          |

| Ribbon section | Command name                            | Parameters |
|----------------|-----------------------------------------|------------|
|                | <a href="#">broadcast-conference</a>    |            |
|                | <a href="#">panel-toggle-conference</a> |            |
|                | <a href="#">panel-toggle-forms</a>      |            |
|                | <a href="#">panel-toggle-links</a>      |            |
|                | <a href="#">panel-toggle-navigation</a> |            |
|                | <a href="#">panel-toggle-bookmarks</a>  |            |

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:

[https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=ribbon\\_commands&rev=1680715445](https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=ribbon_commands&rev=1680715445)

Last update: **2023/04/05 20:24**

