

# Ribbon > Home tab > Viewport group > Apply brightness and contrast

## Button identification




## Associated command

brightness-contrast-dropdown

## Function description

This button brings down a dropdown menu with a number of default settings for brightness and contrast.

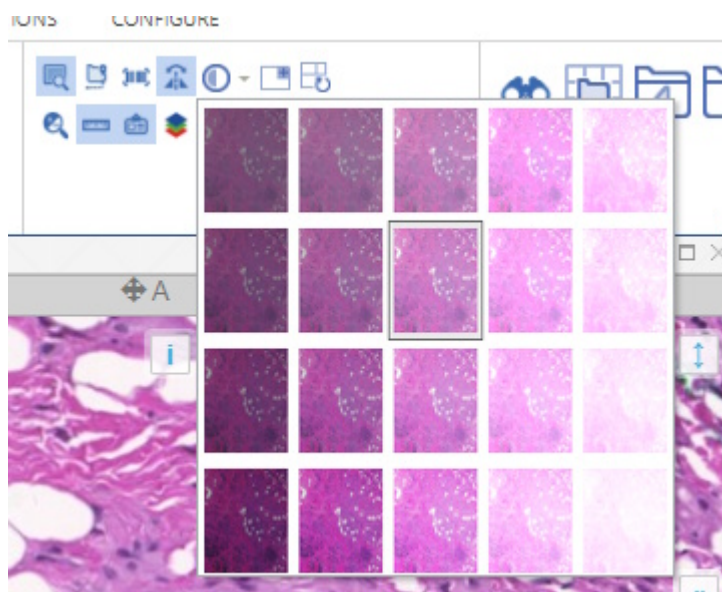

Click the thumbnail to apply the effect that you want. Thumbnails on the left show less brightness, and are brighter on the right. Thumbnails on the top show less contrast, and more contrast on the bottom.

## Related functionality

[Ribbon > Home tab](#)

[Color adjustment panel](#)

Last  
update:  
2023/02/10 ribbon\_home\_viewport\_brightness\_contrast [https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=ribbon\\_home\\_viewport\\_brightness\\_contrast](https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=ribbon_home_viewport_brightness_contrast)  
17:05

---

From:  
<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:  
[https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=ribbon\\_home\\_viewport\\_brightness\\_contrast](https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=ribbon_home_viewport_brightness_contrast)

Last update: **2023/02/10 17:05**

