

Ribbon > Home tab > Viewport group > Reset panel layout

Button identification




Associated command

reset - layout


Function description

This resets the panel layout to the default settings, as defined in the [admin interface](#).

When your layout is like this e.g.:

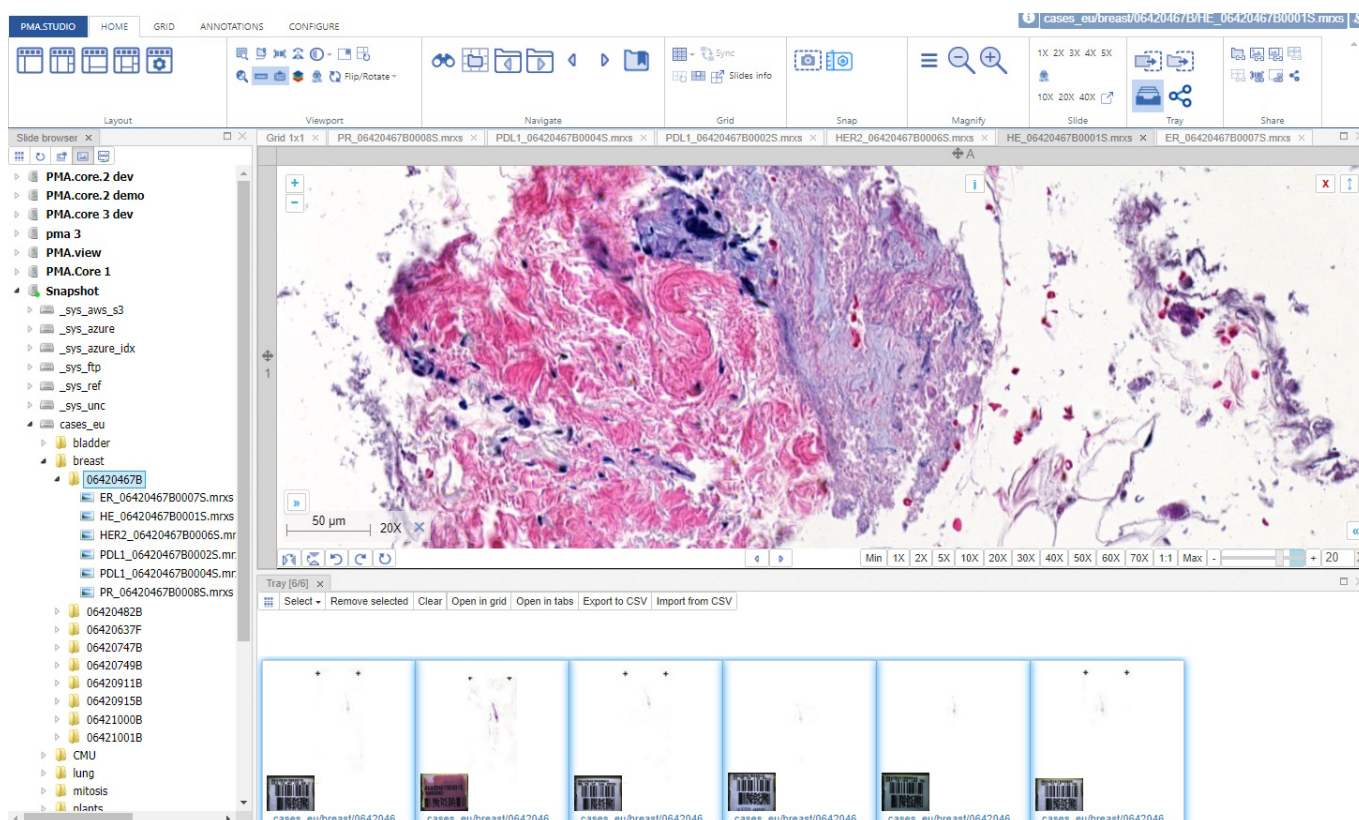

After clicking the button the organization of the panels resets to:

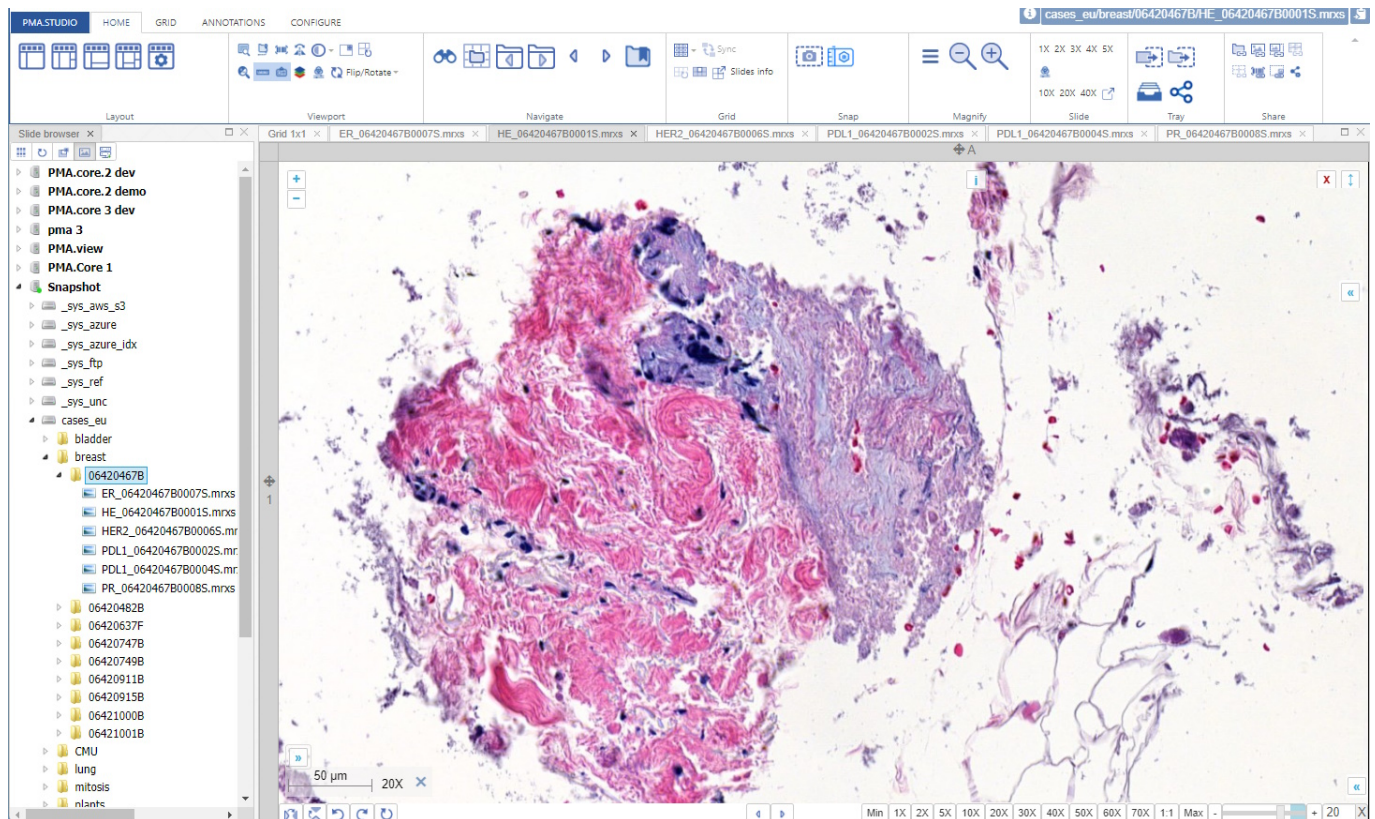

The currently loaded slides are kept.

Also note that the configuration of the panels as stipulated through the admin interface can be different from the one defined by the Default layout button.

Related functionality

[Ribbon > Home tab](#)

[Default layout button](#)

[Administrative interface to define default panel configuration](#)

From:
<https://docs.pathomation.com/pimsls/3.0.0/> - PIMS LS 3.0.0

Permanent link:
https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=ribbon_home_viewport_toggle_reset_panel_layout

Last update: 2023/02/10 18:40

