


## Grid: Size Tool Group

The Size toolgroup contains icons for changing how many cells are visible in your viewport

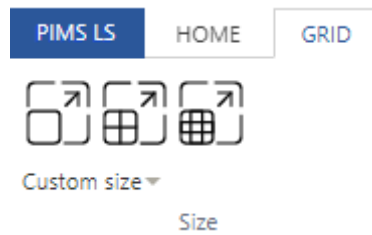

### Top row

In the top row you can see icons for making a 1×1, 2×2, and 3×3 viewport grid, with 1, 4 and 9 images - respectively.

When you click one of the options, the software will ask if you want to keep the current images or replace them.

### Bottom row

The custom size button allows you select how many cells tall and wide you want your viewport to be.

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:

<https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=size&rev=1731675065>

Last update: **2024/11/15 15:51**

