


Sharing Slide(s)


PIMSLS allows you to share content in a variety of ways. There is a dedicated group for sharing content on the [Home](#) and [Grid](#) tabs of the ribbon:

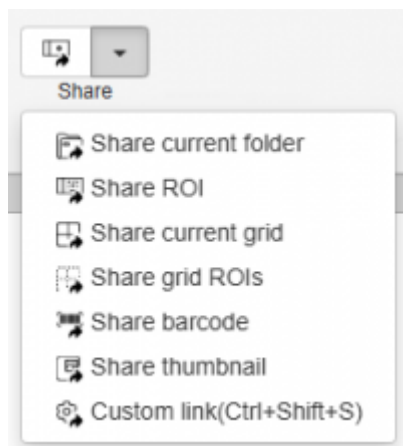

From top left to bottom right the icons are:

1. Quick Share current folder
2. Quick Share current slide
3. Quick Share currently selected ROI / annotation
4. Quick Share current grid
5. Quick Share current ROIs/annotations visible in the grid
6. Quick Share barcode of current slide
7. Quick Share thumbnail of current slide
8. Custom share - this option shows you all the above possibilities and gives you extra customization options

Once you click of any of the Quick Share options you will be shown a popup window that allows you to copy the shared link to clipboard or to scan / download the QR-code

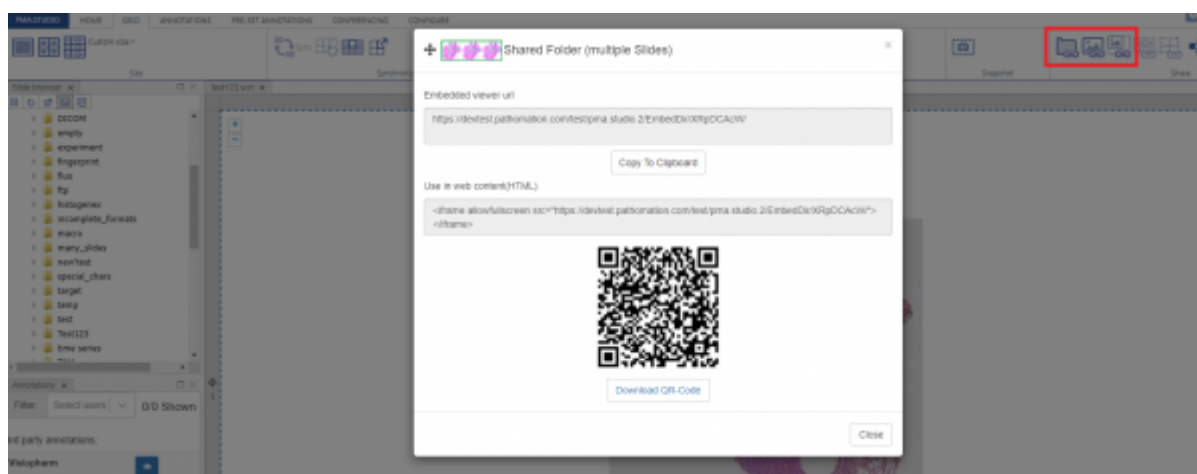

Custom Share

If you want more control over what and how you're sharing content, you can click on the final "Custom Share" button of the group. You could say that that's our "universal" share button.

It allows for further customization of your share link, including:

1. Setting a password to protect your shared content
2. Setting an expiry date (e.g. students can only access it for the duration of a test)
3. Include or exclude the barcode, annotations, and annotation labels from the shared link
4. Give the share a name
5. Add comments

The screenshot shows a 'Share with others' dialog box with a close button (X) in the top right corner. The dialog is divided into two main sections. The top section, titled 'Select what to share', contains a list of sharing options, each with an icon and a corresponding visual preview on the right. The options are: 'Folder (multiple slides)' (icon: folder with arrow, preview: three pink brain icons in a row), 'Slide (single slide)' (icon: single slide with arrow, preview: one pink brain icon), 'Partial slide (ROI)' (icon: slide with a green box, preview: a pink brain icon with a green box), 'Grid of Slides' (icon: 2x2 grid with arrow, preview: a 2x2 grid of pink brain icons), 'Grid of regions of interest' (icon: 2x2 grid with dashed boxes, preview: a 2x2 grid of pink brain icons with green boxes), 'Barcode' (icon: barcode with arrow, preview: a barcode), and 'Thumbnail' (icon: thumbnail with arrow, preview: a small image of a pink brain). The bottom section contains settings: 'Is password protected' (checkbox, unchecked), 'Expires' (checkbox, unchecked), 'Include' (checkboxes for 'Barcode', 'Annotations', and 'Annotations Text', all checked), 'Name' (text input field), and 'Comment' (text input field). At the bottom right are two buttons: 'Link preview' (blue) and 'Cancel' (white with blue border).

Option	Icon	Preview
Folder (multiple slides)	Folder with arrow	Three pink brain icons in a row
Slide (single slide)	Single slide with arrow	One pink brain icon
Partial slide (ROI)	Slide with green box	Pink brain icon with green box
Grid of Slides	2x2 grid with arrow	2x2 grid of pink brain icons
Grid of regions of interest	2x2 grid with dashed boxes	2x2 grid of pink brain icons with green boxes
Barcode	Barcode with arrow	Barcode
Thumbnail	Thumbnail with arrow	Small image of a pink brain

Is password protected ☐

Expires ☐


Include ☒ Barcode ☒ Annotations ☒ Annotations Text

Name


Comment

Link preview Cancel


Password protected preview

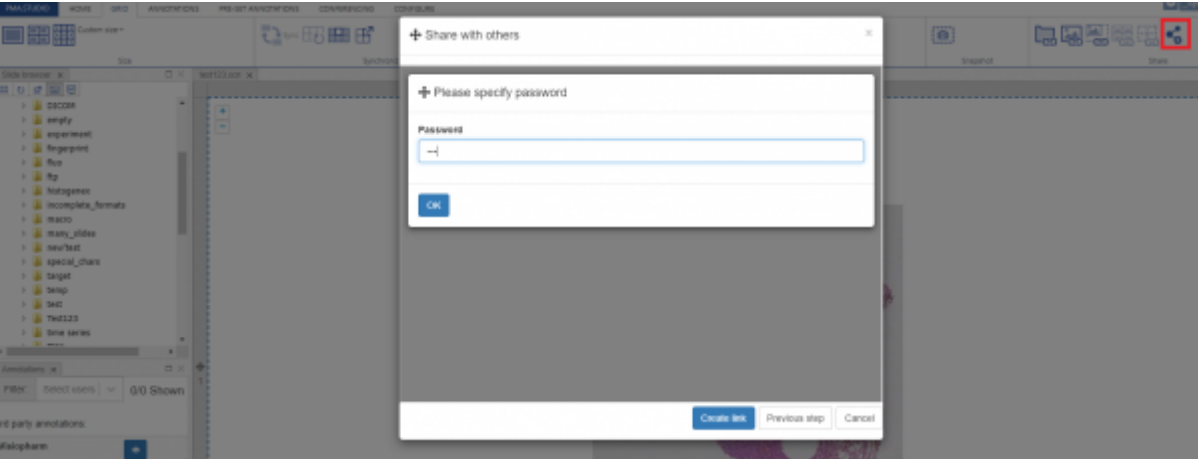

Preview

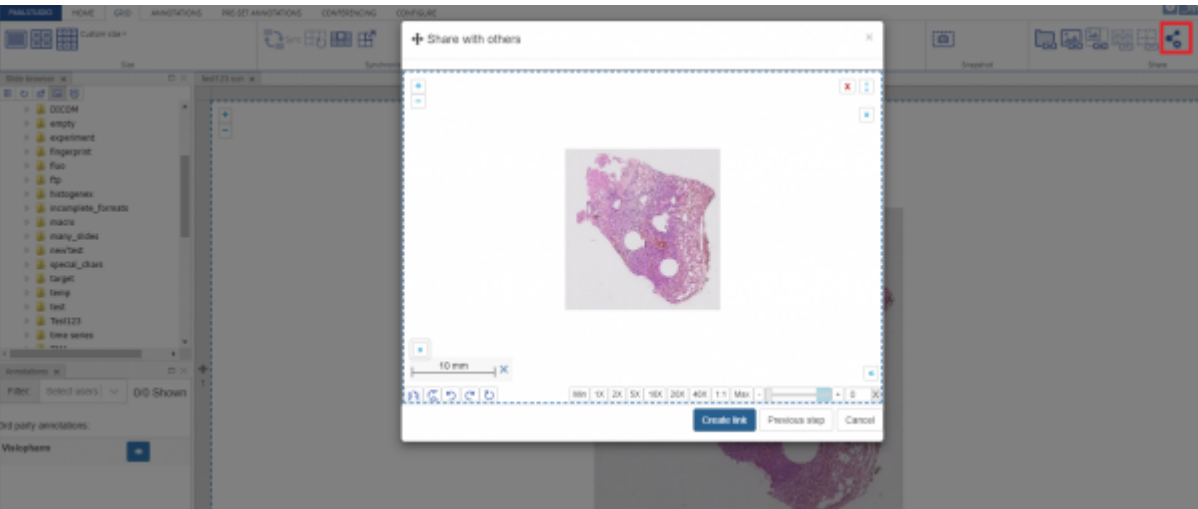

And once you press the “Create Link” button it will generate the share link and show a popup window with option to “Copy to clipboard” or “Download QR-Code”

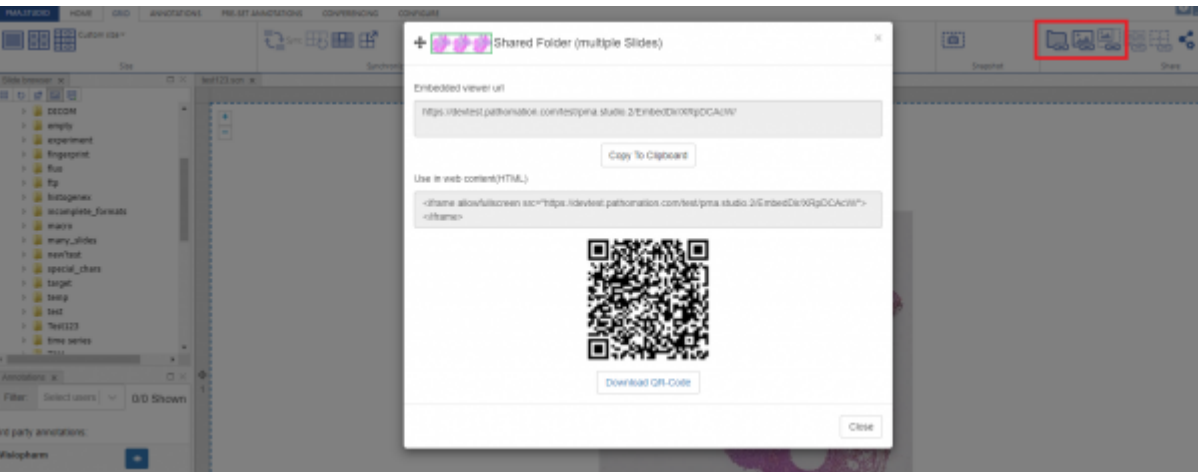

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - **PIMS LS 3.0.0**

Permanent link:

https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=slides_sharing&rev=1746902039

Last update: **2025/05/10 21:33**

