


Annotations: Style Workgroup

The Annotation style toolgroup contains icons for configuring annotation colour and transparency.

All annotation styles are set by clicking the down arrow and choosing a colour, number, or percentage from the dropdown menu.


Annotation colour

Clicking the annotation colour lets you choose the outline colour of your annotation

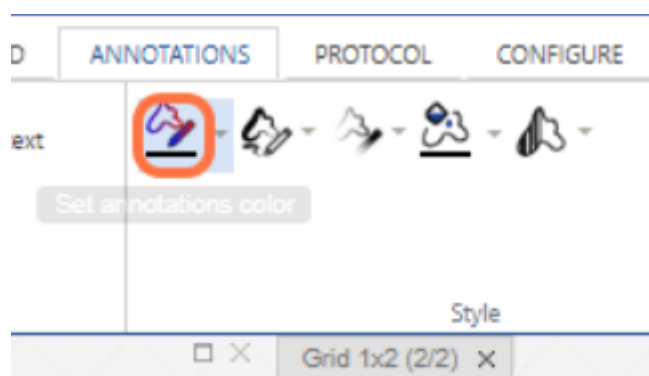


Annotation thickness

Clicking the annotation thickness lets you choose the thickness of the line of your annotation

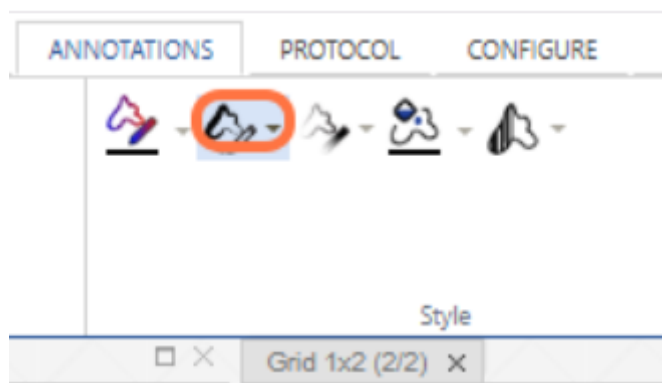


You can choose a number from 1-10 in the dropdown menu:

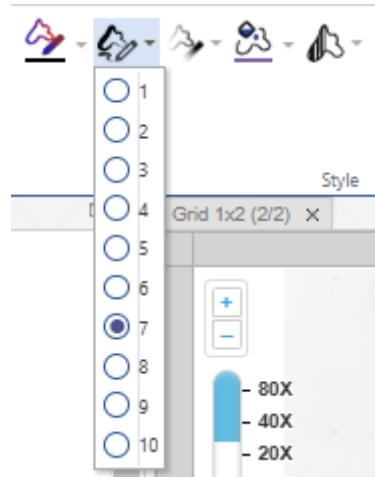


Annotation opacity

Clicking the annotation opacity lets you choose how opaque/transparent your annotation outline is

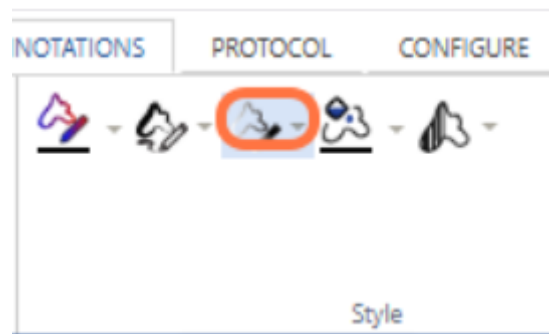


Fill colour

Clicking the fill colour lets you choose the fill colour of your annotation (if its a closed shape)

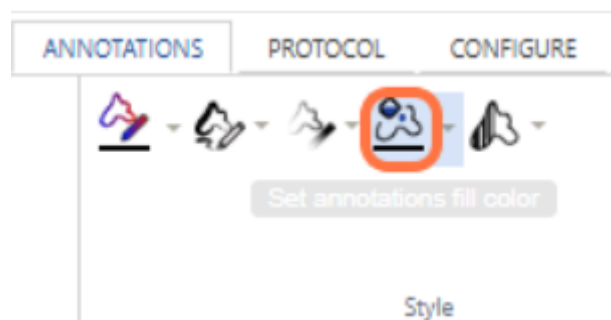


You can select a colour from the dropdown menu:

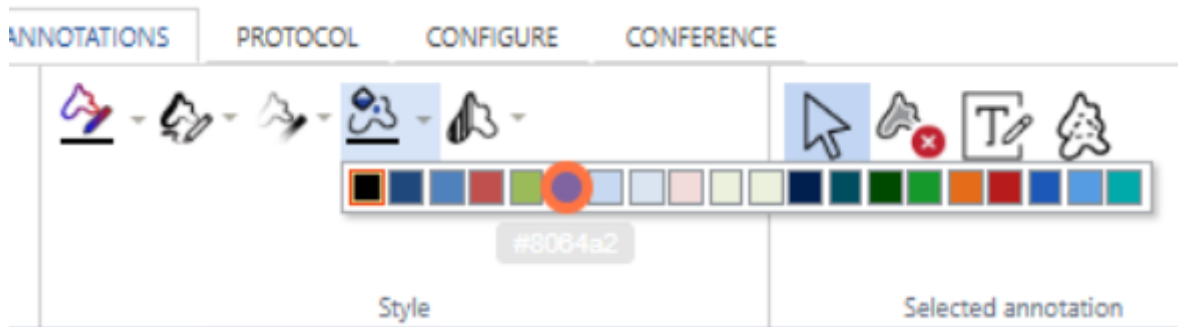

Fill opacity

Clicking the fill opacity lets you choose the fill opacity/transparency for your closed shape annotations e.g. square, closed freehand

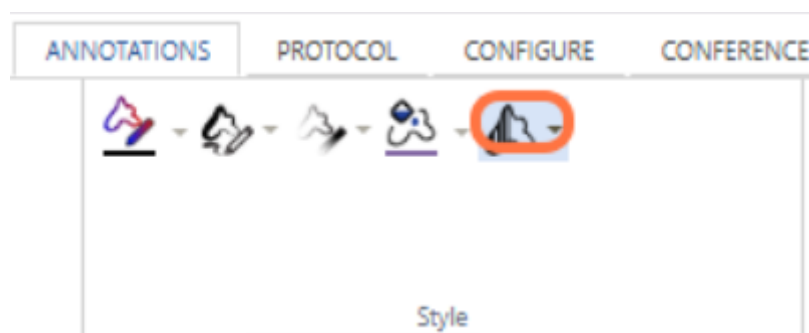

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - PIMS LS 3.0.0

Permanent link:

<https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=style&rev=1732807823>

Last update: 2024/11/28 18:30

