


# Annotations: Style Workgroup

The Annotation style section contains icons for configuring annotation colour and transparency.

All annotation styles are set by clicking the icon and choosing a colour, number, or percentage from the dropdown menu.


## Annotation colour

Clicking the annotation colour lets you choose the outline colour of your annotation

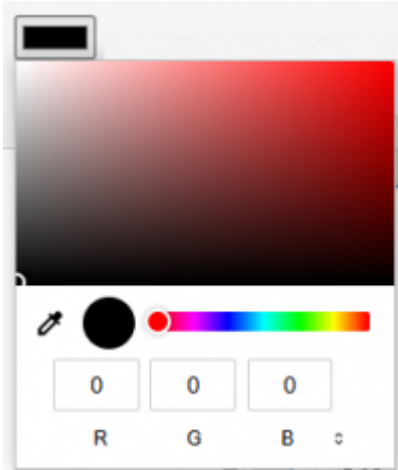


## Annotation thickness

Clicking the annotation thickness lets you choose the thickness of the line of your annotation

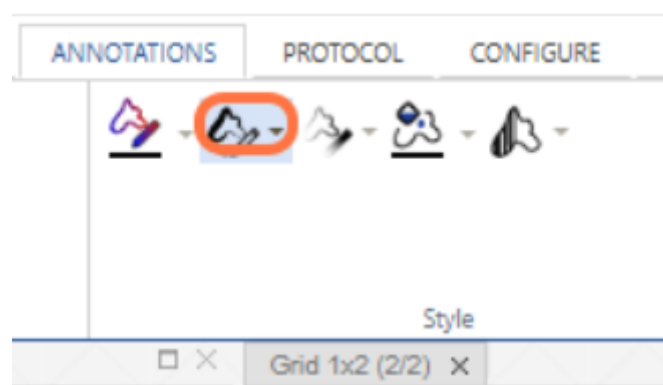


You can choose a number from 1-10 in the dropdown menu:

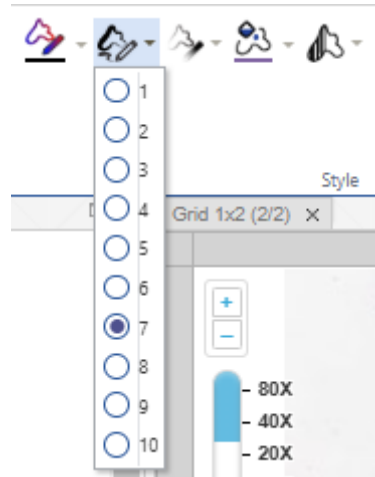


## Annotation opacity

Clicking the annotation opacity lets you choose how opaque/transparent your annotation outline is

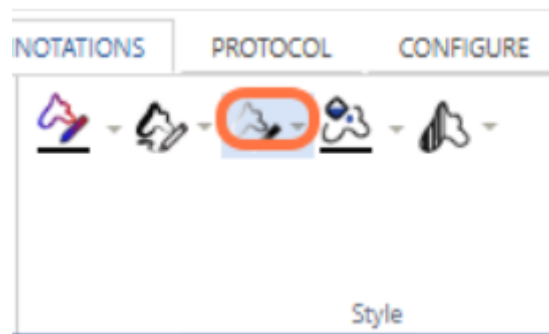


## Fill colour

Clicking the fill colour lets you choose the fill colour of your annotation (if its a closed shape)

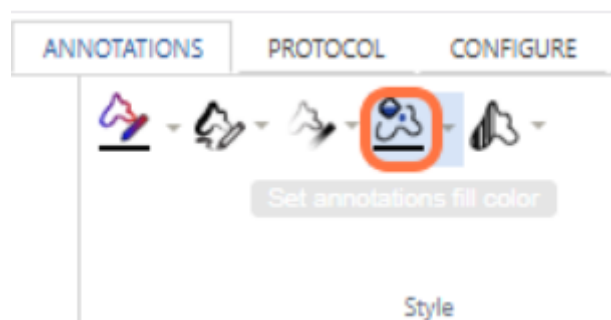


You can select a colour from the dropdown menu:

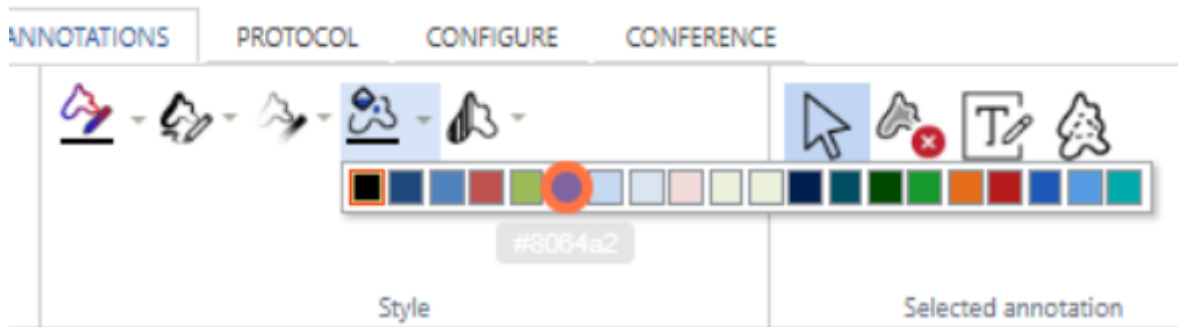

## Fill opacity

Clicking the fill opacity lets you choose the fill opacity/transparency for your closed shape annotations e.g. square, closed freehand

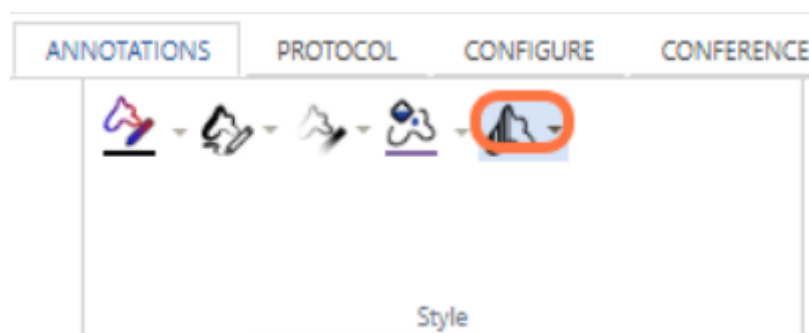

From:

<https://docs.pathomation.com/pimsls/3.0.0/> - PIMS LS 3.0.0

Permanent link:

<https://docs.pathomation.com/pimsls/3.0.0/doku.php?id=style&rev=1746908360>

Last update: 2025/05/10 23:19

