


# Ribbon layout (administration)

This admin page allows you to setup default Ribbon layout that consists of different tabs and controls. Each tab consists of different controls associated to it and you can configure appearance of tabs and controls using this page.

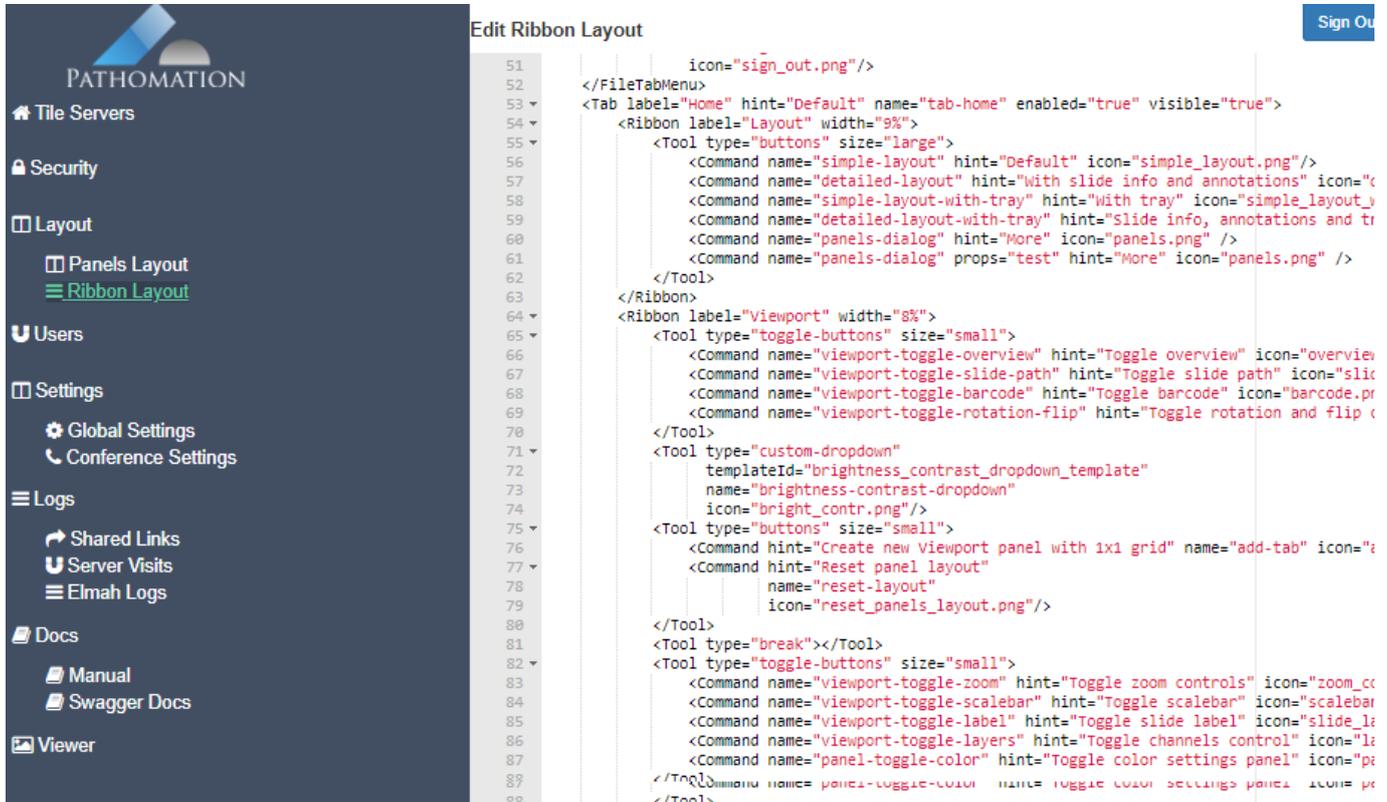

The screenshot shows the 'Edit Ribbon Layout' page in the Pathomation administration interface. The left sidebar contains a navigation menu with the following items: Tile Servers, Security, Layout (with sub-items Panels Layout and Ribbon Layout), Users, Settings (with sub-items Global Settings and Conference Settings), Logs, Shared Links, Server Visits, Elmah Logs, Docs (with sub-items Manual and Swagger Docs), and Viewer. The main content area displays the XML code for configuring the ribbon layout, including tabs like 'Home' and 'Layout', and various tools and commands.

An [exhaustive list of all available commands](#) is available elsewhere on this site.

Below are some of the tabs and corresponding controls used in PMA.Studio.

**Home tab** has 8 sections:

1. View
2. Grid
3. Snap
4. Magnify
5. Directory
6. Slide
7. Share
8. Search



**Grid tab** supplies instruments to manage grid and it has 6 sections:

1. ItemSize
2. Synchronize
3. Rows/Columns
4. Slide

- 5. Snapshot
- 6. Share



**Annotations tab (available in PMA.Studio)** supplies instruments for working with annotations and has 4 sections:

- 1. Annotations
- 2. Shapes
- 3. Export
- 4. Import




**Pre-set Annotations tab (available in PMA.Studio)** consists of 3 sections:

- 1. Annotations
- 2. Stamps
- 3. Protocols




**Conferencing tab** has 2 sections:

- 1. Conferencing
- 2. Server

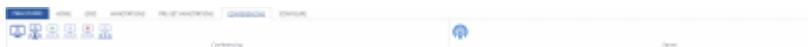

**Configure tab** has 4 sections:

- 1. Server
- 2. Panels
- 3. Pre-set layouts
- 4. Viewport

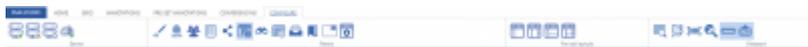

From:  
<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:  
[https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=admin\\_ribbon&rev=1674470758](https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=admin_ribbon&rev=1674470758)

Last update: **2023/01/23 13:45**

