

Tile servers (administration)

In this section of PMA.studio's administrative portal, you can manage the different PMA.core instances that are used by PMA.studio to retrieve content from.

PMA.studio (PMA.view) does not function without connectivity to a PMA.core tile server. If you want to learn more about what a tile server is, [we have a YouTube video on this topic](#).

This page allows you to add different types of servers used by PMA.Studio including PMA.core, PMA.search, PMA.start, My Pathomation and PMA.live. You can press "Add New" button to add a new server.

The screenshot shows the 'Create new server' form with the following fields and options:

- Server Type:** PMA.core (selected), PMA.start, My Pathomation, PMA.core, PMA.live
- Name:** (text input)
- URL:** (text input)
- Internal URL:** (text input)
- PMA Search server URL:** (text input)
- PMA Search server token:** (text input)
- Use internal URL:** (checkbox)
- Username:** (text input)
- Password:** (text input)
- Buttons:** Add New, Cancel

It allows you to manage the list of created servers where you can view server status (online/offline) along with setting the order of these servers to appear in PMA.Studio viewer. Besides you can delete a server as per your convenience using the delete button.

Name	URL	Internal URL	Status	Server Version	Order	Action
PMA Core 1	http://pma.pathomation.com/8080/nextgen/next/1/		Online	1.0.0.0.0.0	1	[Up] [Down] [Delete]
Search PMA Core 2	https://search.pathomation.com/nextgen/pma/core/2/		Online	2.0.0.0.0.0	2	[Up] [Down] [Delete]
Start	https://start.pathomation.com/pma/core/3/		Online	2.0.0.0.0.0	3	[Up] [Down] [Delete]
My Pathomation	https://my.pathomation.com/next/		Offline		4	[Up] [Down] [Delete]
Live	https://live.pathomation.com/next/		Offline		5	[Up] [Down] [Delete]
Live	https://live.pathomation.com/next/		Offline		6	[Up] [Down] [Delete]

From:
<https://docs.pathomation.com/pimsls/3.0.1/> - PIMS LS 3.0.1

Permanent link:
https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=admin_tileservers&rev=1648660513

Last update: 2022/03/30 20:15

