

Tile servers (administration)

In this section of PMA.studio's administrative portal, you can manage the different PMA.core instances that are used by PMA.studio to retrieve content from.

PMA.studio (PMA.vue) does not function without connectivity to a PMA.core tile server. If you want to learn more about what a tile server is, [we have a YouTube video on this topic](#).

This page allows you to add different types of servers used by PMA.Studio including [PMA.core](#), [PMA.search](#), [PMA.start](#), [My Pathomation](#) and [PMA.live](#). You can press "Add New" button to add a new server.

The screenshot shows the 'Create new server' form with the following fields and options:

- Server Type:** PMA.core (selected), PMA.start, My Pathomation, PMA.search, PMA.live
- Name:** Text input field
- URL:** Text input field
- Interpret URL:** Radio button (selected), or PMA.core URL, or PMA.start URL, or My Pathomation URL, or PMA.search URL, or PMA.live URL
- PMA Search server url:** Text input field
- PMA Search server id:** Text input field
- Use interpretation:** Radio button (selected), or PMA.core URL, or PMA.start URL, or My Pathomation URL, or PMA.search URL, or PMA.live URL
- Username:** Text input field
- Password:** Text input field
- Interpretation:** Text input field
- Buttons:** Add, Cancel

It allows you to manage the list of created servers where you can view server status (online/offline) along with setting the order of these servers to appear in PMA.Studio viewer. Besides you can delete a server as per your convenience using the delete button.

Name	URL	Interpret URL	Status	Server Version	Order	Action
PMA.core 1	http://pma.pathomation.com/8080/pma/core/1/		Online	1.0.0.0.0	1	[Up] [Down] [Delete]
Custom PMA.core 2	https://custom.pathomation.com/8080/pma/core/2/		Online	1.0.0.0.0	2	[Up] [Down] [Delete]
PMA	https://pma.pathomation.com/pma/core/3/		Online	1.0.0.0.0	3	[Up] [Down] [Delete]
Interpretation	https://pma.pathomation.com/pma/core/4/		Offline		4	[Up] [Down] [Delete]
URL	https://pma.pathomation.com/pma/core/5/		Offline		5	[Up] [Down] [Delete]
URL	https://pma.pathomation.com/pma/core/6/		Offline		6	[Up] [Down] [Delete]

Notes on PMA.live

More background on PMA.live can be found at <https://www.pathomation.com/pma.live>.

From:
<https://docs.pathomation.com/pimsls/3.0.1/> - PIMS LS 3.0.1

Permanent link:
https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=admin_tileservers&rev=1674040505

Last update: 2023/01/18 14:15

