


## Annotations import and export

PIMS LS allows you to import and export annotations in a variety of formats, including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

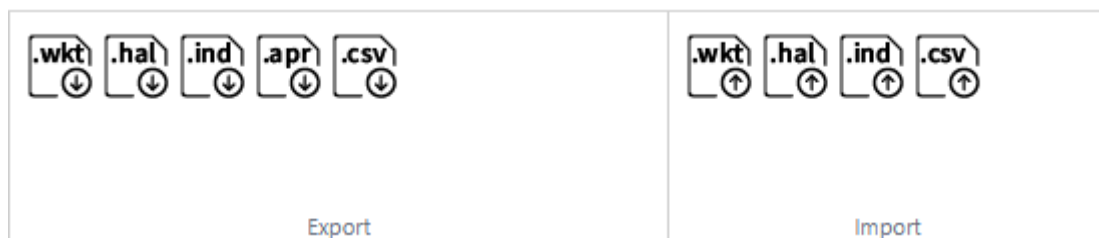


### Export

You can export slide annotations in different formats including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

### Import

User can also import/upload annotations through PIMS LS. Once you click on the import button for the format you want to import, a menu will appear prompting you to select the file that contains annotation data.

Once a valid file is uploaded annotations are added to the slide.

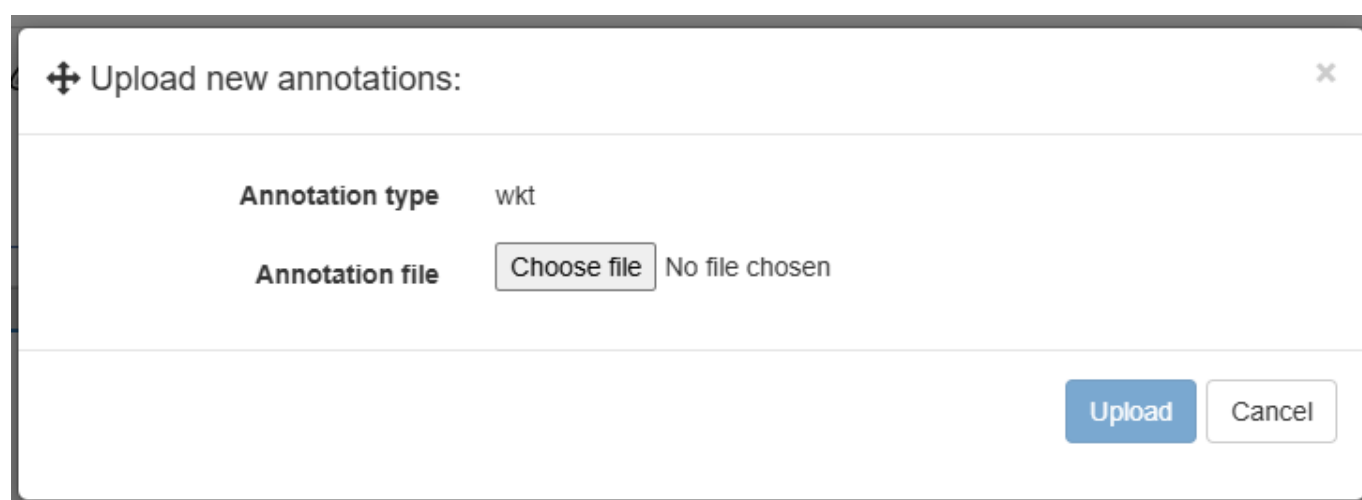

Last  
update: 2024/11/29 11:38 annotations\_import\_export [https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=annotations\\_import\\_export&rev=1732869496](https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=annotations_import_export&rev=1732869496)

---

From:  
<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:  
[https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=annotations\\_import\\_export&rev=1732869496](https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=annotations_import_export&rev=1732869496)

Last update: **2024/11/29 11:38**

