


Configure Server

The Server toolgroup in the configure tab contains everything you need to connect to different kinds of storage.

Not only can you connect to other PMA.Cores, but also to storage on Pathomation's cloud platform [My Pathomation](#) if you have an account - FYI your first 10 GB is free!

You can also connect to [PMA.start](#), our desktop application for local storage connection.

Connecting to another PMA.core

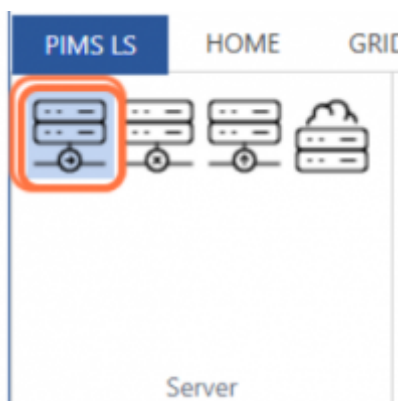

Clicking the first icon will open the menu for connecting to another PMA.core, pictured below:

A screenshot of a dialog box titled 'Quick connect to Tile server'. The dialog box has a close button (X) in the top right corner. Inside the dialog, there are four input fields: 'Server Name' (placeholder: Server Name), 'Server Url' (placeholder: Tile Server Url), 'Username' (placeholder: Username), and 'Password' (placeholder: Password). The 'Username' and 'Password' fields have red dots indicating they are required. Below the input fields, there is a 'Save' checkbox with an information icon. At the bottom of the dialog, there are three buttons: 'Check Connection', 'Add Server', and 'Reset'. A 'Close' button is located at the bottom right of the dialog box.

Here you can input:

1. Server name: this is a nickname for the Core/Drive you want to connect to
2. Server URL: this is the web address of the PMA.core you want to connect to e.g. [mycore.mybusiness.com/core/](#)


3. Username: Your username for logging into the PMA.core
4. Password: Your password for logging into the PMA.core

You can check the connection to see if all the details work.


If you need to be connected to a VPN to access your PMA.core, ensure that you are connected to the right VPN.

Disconnect Server

Clicking this button will disconnect you from all PMA.core servers you have manually connected. Please note that any servers that are set to be automatically logged into will not be affected.

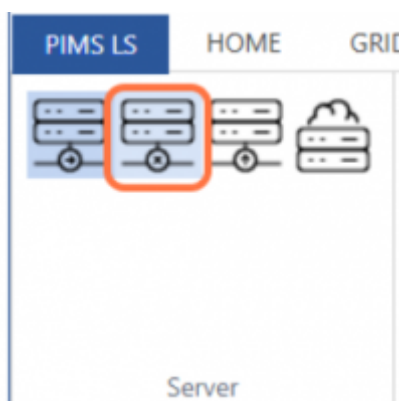


Connect to PMA.start

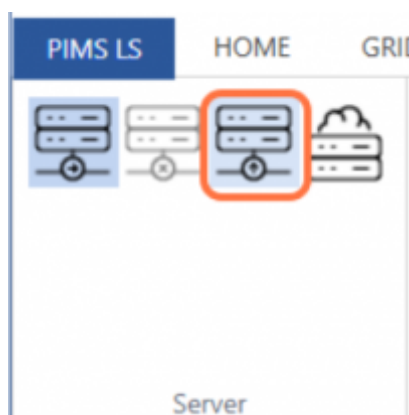


The third icon will search for an active instance of PMA.start on your device and integrate it in your slide browser


Please note - ensure that PMA.start is running before clicking the button

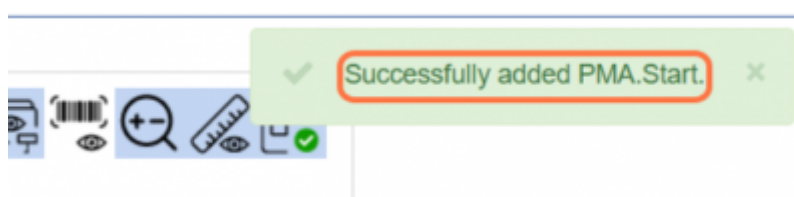

Connect to My Pathomation

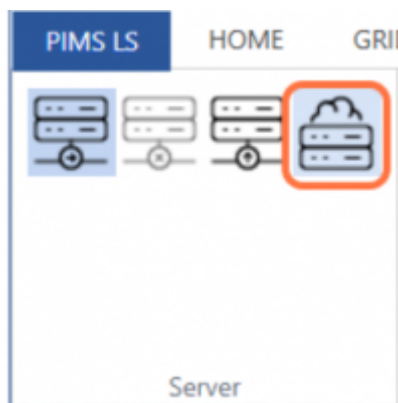

You can also connect to your MyPathomation cloud account
Simply login with your email address and password as you would on my.pathomation.com and your content will seamlessly be added.

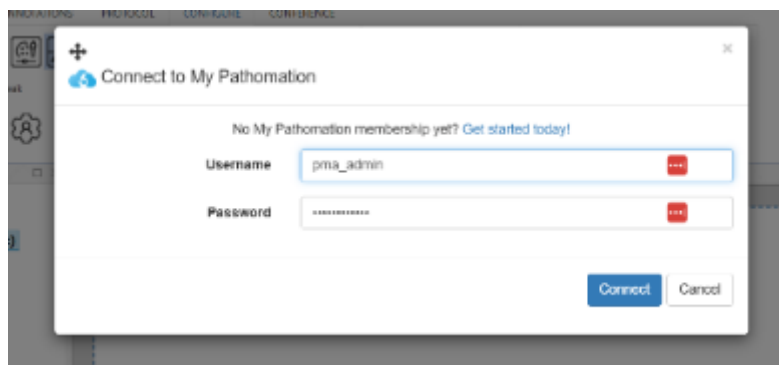

From:

<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:

<https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=configserver>

Last update: **2024/12/04 16:01**

