


# Home: Grid Tool

The Grid tool contains icons for synchronising different viewports in the grid, clearing the grid, and opening new grid tabs.

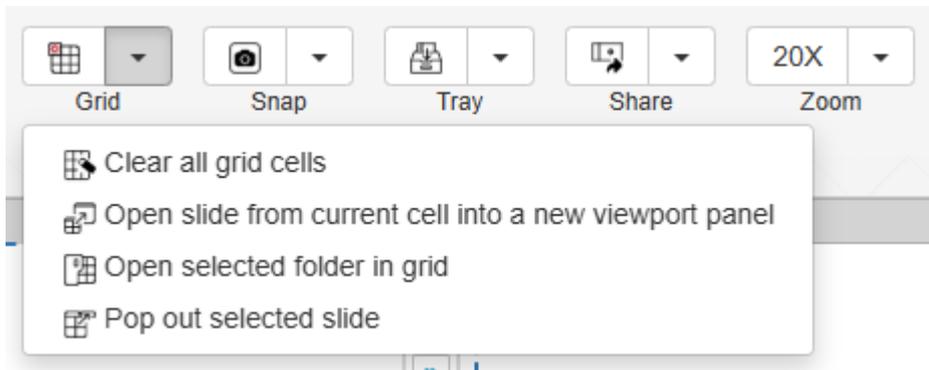

- Clear all grid cells - clears all images from current grid
- Opens current folder in grid - opens the menu below

**+** Open in Grid ✕

You are about to open 4 slides in a grid

**Slides in current grid** 1

<input checked="" type="checkbox"/>	Slide	Thumbnail
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1.mrxs</code>	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1.ndpi</code>	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1.svs</code>	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1-JP2K-33005.svs</code>	

How do you want to do it?

- Opens slide from current cell into a new viewport panel
- Pops out selected slide in a new window.
- Clear current cell - clears image from currently selected grid cell

From:

<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:

<https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=grid&rev=1746901236>

Last update: **2025/05/10 21:20**

