


Home: Grid Tool

The Grid dropdown contains tools for clearing the grid, opening new grid tabs, open current folder in grid and popping out selected slide in new window.

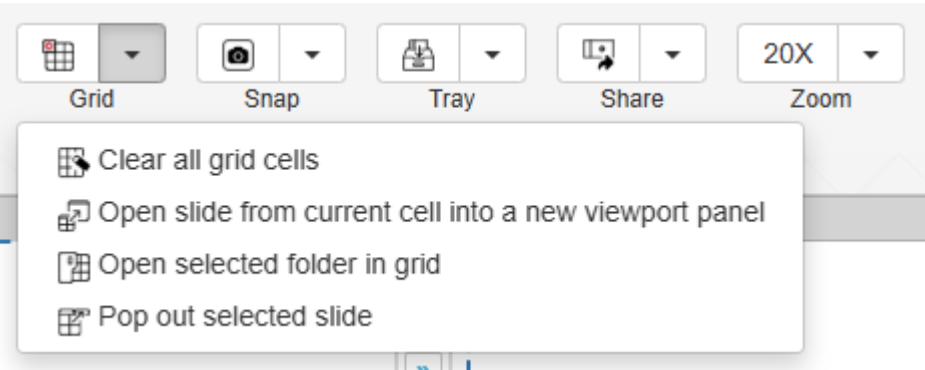

- Clear all grid cells - clears all images from current grid
- Opens current folder in grid - opens the menu below

Open in Grid

You are about to open 4 slides in a grid

Slides in current grid

1

<input checked="" type="checkbox"/>	Slide	Thumbnail
<input checked="" type="checkbox"/>	<div>_sys_ref/annotations</div> <div>CMU-1.mrxs</div>	
<input checked="" type="checkbox"/>	<div>_sys_ref/annotations</div> <div>CMU-1.ndpi</div>	
<input checked="" type="checkbox"/>	<div>_sys_ref/annotations</div> <div>CMU-1.svs</div>	
<input checked="" type="checkbox"/>	<div>_sys_ref/annotations</div> <div>CMU-1-JP2K-33005.svs</div>	

How do you want to do it?

Clear current grid and open

Append to current grid

Open in new grid

Cancel

- Opens slide from current cell into a new viewport panel
- Pops out selected slide in a new window.
- Clear current cell - clears image from currently selected grid cell

From:

<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:

<https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=grid&rev=1747058920>

Last update: **2025/05/12 17:08**

