

Keyboard Shortcuts

Below you can find a list of Keyboard Shortcuts.

Quick-Connect to server

- ctrl + q

Save snapshot

- ctrl + s

Share

- ctrl + shift + s

Toggle multi-selection mode

- ctrl + m

Decrease zoom

- [

Increase zoom

-]

Select next image

- ctrl + down arrow

Select previous image

- ctrl + up arrow

Select next directory

- ctrl + shift + down arrow

Select previous directory

- ctrl + shift + up arrow

Export Annotations

- ctrl + shift + a

Zoom into area

- shift + click and drag box over slide area

Rotate slide

- alt + click & drag

From:

<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:

https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=keyboard_shortcuts

Last update: **2024/12/17 11:54**

