

# Annotations

PMA.studio supports several ways of creating and managing annotations.

When opening a slide that has annotations, and the panel is not open yet, a prompt will ask if you want to activate the panel:




Access to PMA.studio's annotation functions is handled via a dedicated "Annotations" tab on the ribbon:



The first thing you probably want to do when working with annotations is to enable the annotations panel by clicking the "Toggle annotations panel" button:



Once you click it, you will see screen like this:

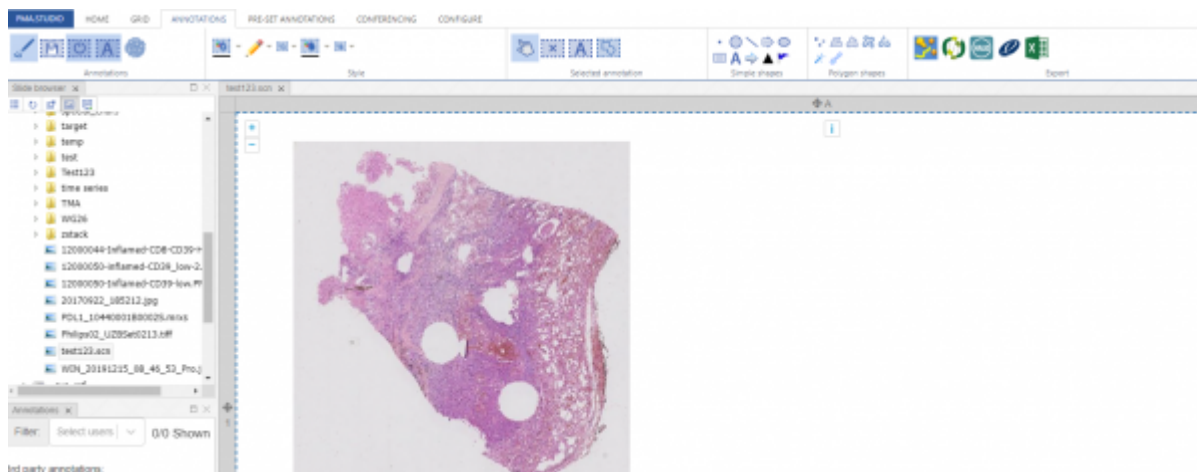

So, that was just the beginning and we have not many any actual annotation as yet. Now, if we look at the "Annotation" tab content on the ribbon it consists of different types of annotations tools we can use to draw annotations on a selected slide.

PMA.Studio provides different set of annotations tools you can choose from and draw annotations:

## (Simple shapes)

1. Point
2. Circle
3. Line
4. Arrow

5. Ellipse
6. Rectangle
7. Text
8. Landmark 1
9. Landmark 2
10. Landmark 3

## (Polygon Shapes)

1. Multipoint
2. Polygon
3. Closed Freehand
4. Compound Freehand
5. Freehand
6. Wand
7. Brush



From:  
<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:  
[https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=panel\\_annotations&rev=1674039931](https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=panel_annotations&rev=1674039931)

Last update: **2023/01/18 14:05**

