

# Slide information panel

The Info panel displays available slide information for currently selected slide.

Info	
File name	_sys_ref/brightfield/Grundium/hp1 (1).svs
Last Modified	Mar 01 2021 18:11
Width	131,072 pixels / 32.840 mm
Height	73,728 pixels / 18.473 mm
Max Magnification	40X
Micrometres Per Pixel X	0.250552
Micrometres Per Pixel Y	0.250552
Physical Size	328.53 MB
Tile Size	512 x 512 pixels
Barcode	
Barcode text	
Overview	
Macro	N/A
Layers	1
Channels	1

Currently selected slide is highlighted with blue dashed border. Columns are pretty self-descriptive and only thing to mention here is that barcode and overview is clickable and by clicking on that images it will be opened in a new browser tab.

From: <https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link: [https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=panel\\_info&rev=1677836723](https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=panel_info&rev=1677836723)

Last update: **2023/03/03 12:45**

