


corner of the screen, too. There are plus and minus button to zoom in (increase magnification) or zoom out (reduce magnification).




There is also the zoombar. You interact with that one by moving the slider up and down to control your magnification.

The zoombar is useful even if you don't use it to control your magnification: it also indicates whether you are looking at content through optimal magnification or digital pmagnificaion. The number of artificiofically added zoomlevels for digital zoom can be controlled in PMA.UI, or even disabled altogether.

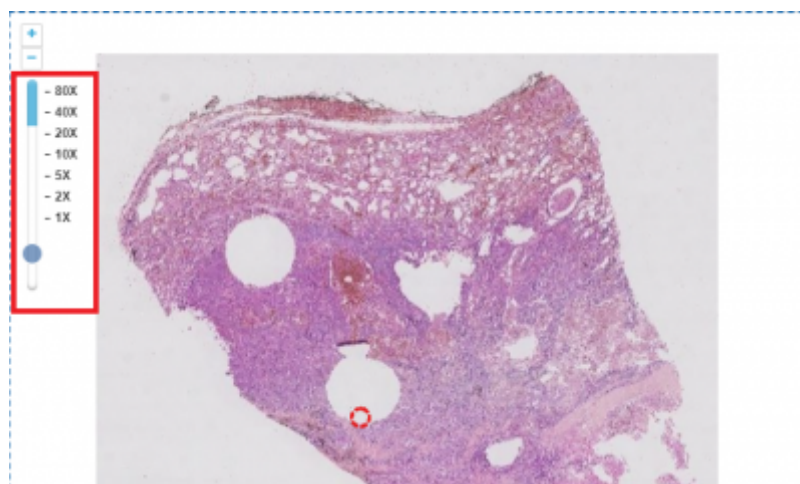

Thumbnails and labels

Associated images are images that are part of the whole slide image data package, and provide additional information about the slide. Examples include a thumbnail image, a scan map, barcode image, or a label image. The first and the latter of this list are automatically added to the right of the viewport, if they are available.

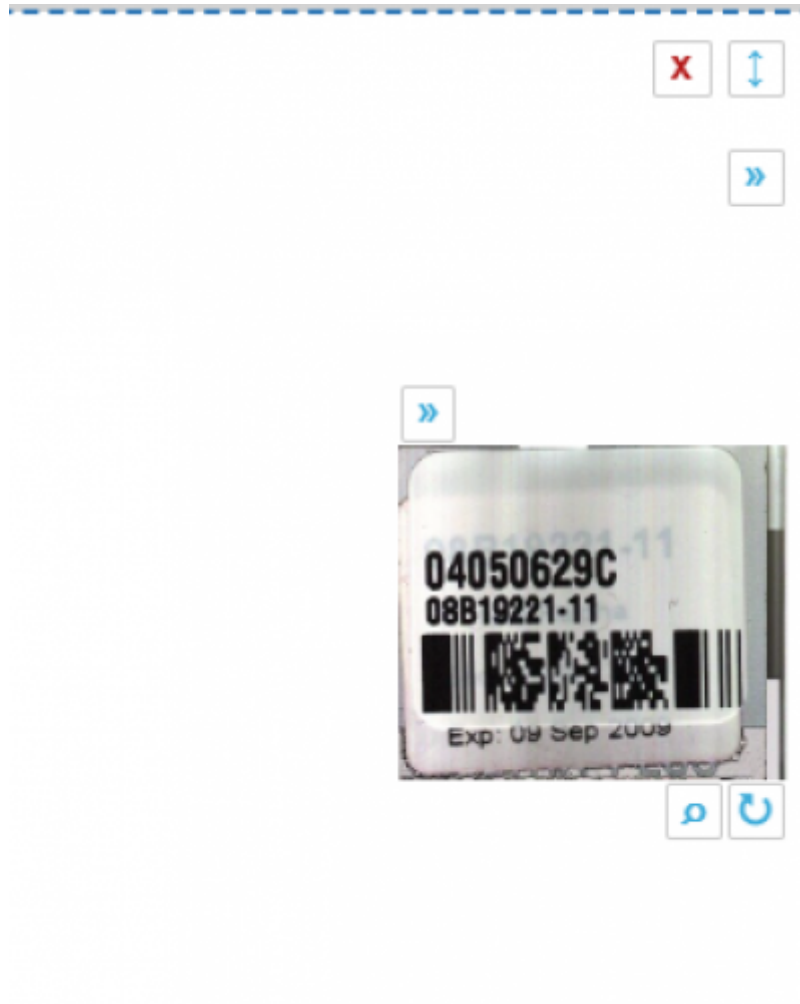

From:
<https://docs.pathomation.com/pimsls/3.0.1/> - PIMS LS 3.0.1

Permanent link:
https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=panel_viewport&rev=1651782358

Last update: **2022/05/05 23:25**

