

Available ribbon commands

Ribbon section	Command name	Parameters
File menu	custom-panel-algo-ichi	props=
	custom-panel-algo-ni	props=
	custom-panel-ai-prov-mindpeak	props=
	custom-panel-ai-prov-spintellx	props=
	file	
	about-us	
	show-eula	
	manual	
	sign-out	
Ribbon section	Command name	Parameters
Home tab	simple-layout	
	detailed-layout	
	detailed-layout	
	viewport-toggle-overview	
	viewport-toggle-slide-path	
	viewport-toggle-barcode	
	add-tab	
	reset-layout	
	viewport-toggle-zoom	
	viewport-toggle-scalebar	
	viewport-toggle-label	
	panel-toggle-color	
	panel-toggle-search	
	open-folder-in-grid	
	prev-directory	
	next-directory	
	prev-image	
	next-image	
	add-folder-bookmark	
	reset-grid	
	clear-cell	
	open-in-tab	
	popout-info	
	snapshot	
	quick-snapshot	
	toggle-toolbar	
	increase-zoom	
	decrease-zoom	
	custom-zoom-1x	pma-magnification=
	custom-zoom-2x	pma-magnification=
	custom-zoom-5x	pma-magnification=
	custom-zoom-10x	pma-magnification=
	custom-zoom-20x	pma-magnification=

Ribbon section	Command name	Parameters
	custom-zoom-40x	pma-magnification=
	panel-toggle-color	
	send-slide-to-tray	
	send-to-tray	
	panel-toggle-tray	
	share-slide-tray	
	quick-share-folder	
	quick-share-slide	
	quick-share-slide-roi	
	quick-share-grid	
	quick-share-grid-rois	
	quick-share-thumbnail	
	share-slide	
Ribbon section	Command name	Parameters
Grid tab	toggle-1x1-grid	
	toggle-2x2-grid	
	toggle-3x3-grid	
	sync-views-toggle	
	reset-grid	
	clear-cell	
	open-in-tab	
	viewport-add-row	
	viewport-remove-row	
	viewport-remove-row	
	viewport-remove-column	
	flip-rotate-menu	
	reset-flip	
	flip-horizontally	
	flip-vertically	
	reset-rotation	
	rotate-clockwise-90	
	rotate-counter-clockwise-90	
	rotate-180	
	zoom-in	
	panel-toggle-annotations	
	save-annotations	
	toggle-annotations	
	toggle-annotation-text	
	toggle-fingerprint	
Ribbon section	Command name	Parameters
Annotations tab	Cursor	
	delete-annotation	
	delete-all-annotations	
	edit-annotation-text	
	merge-selected-annotations	

Ribbon section	Command name	Parameters
	modify-selected-annotation	
	Point	
	Circle	
	Line	
	Arrow	
	Ellipse	
	Rectangle	
	Landmark1	
	MultiPoint	
	Polygon	
	ClosedFreehand	
	CompoundFreehand	
	Freehand	
	Wand	
	Brush	
	export-to-wkt	
	export-to-mld	
	export-to-annotations	
	export-to-aperio	
	export-to-csv	
	import-to-wkt	
	import-to-mld	
	import-to-halo	
	import-to-csv	
Ribbon section	Command name	Parameters
Pre-set tab	preset-Circle0237mm2	pma-shape, pma-size-a, pma-size-b
	preset-Circle1mm	pma-shape, pma-size-a, pma-size-b
	preset-Circle2mm	pma-shape, pma-size-a, pma-size-b
	preset-Circle1cm	pma-shape, pma-size-a, pma-size-b
	preset-Square5mm	pma-shape, pma-size-a, pma-size-b
	preset-Square1cm	pma-shape, pma-size-a, pma-size-b
	preset-Tumor	pma-classification, pma-shape, pma-color
	preset-Macro	pma-classification, pma-shape, pma-color
Ribbon section	Command name	Parameters
Conferencing tab	create-conference	
	join-conference	
	start-conference	
	pause-conference	
	stop-conference	
	toggle-everyone-control-conference	
	broadcast-conference	
	panel-toggle-conference	
	panel-toggle-forms	
	panel-toggle-links	
	panel-toggle-navigation	

Last update: 2023/01/18
13:42

ribbon_commands https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=ribbon_commands&rev=1674038566

Ribbon section	Command name	Parameters
	panel-toggle-bookmarks	

From:

<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:

https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=ribbon_commands&rev=1674038566

Last update: **2023/01/18 13:42**

