


Working with grids

PMA.studio uses viewport as a grid system. UI provides a separate tab “GRID” to perform grid operations on the viewport.

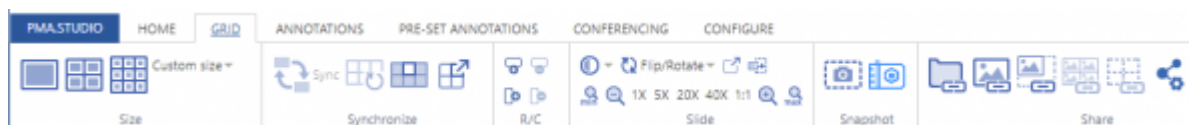


“GRID” tab consists of 6 sections:

1. [Size](#)
2. [Synchronize](#)
3. [R/C](#) (rows and columns)
4. [Slide viewing properties](#)
5. [Snapshot](#)
6. [Share](#)


Size

Size section allows you to break viewport as a grid system with different sizes. This includes grid sizes of 1×1, 2×2, 3×3 and even you can customized the grid size according to your need.

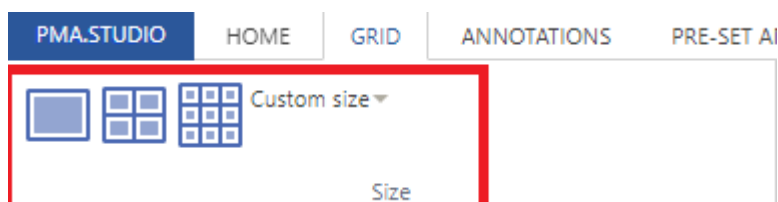

(1×1 Grid size)

First grid option allows you to divide the viewport into 1×1 column grid.

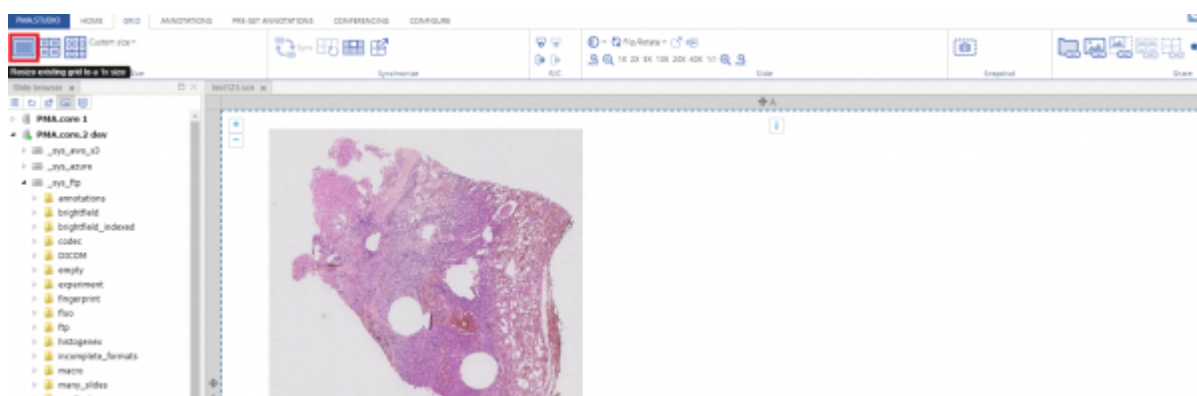

(2×2 Grid size)


Second grid option divides the viewport into 2×2 column grid.



(Custom Grid size)

Synchronize

This section allows you to perform different actions as a whole on individual grid column or as a whole grid system. It is further divided into 4 sub actions:

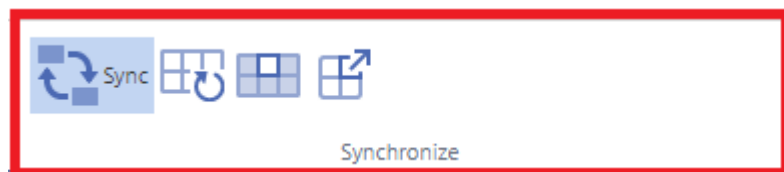

Toggle synchronous navigation between cells in the current grid

This button allows you to perform set of synchronous actions on whole grid.

Clear all grid cells


This button allows clearing all the grid cells at the same time

Clear current cell only

This button allows to clear the selected grid cell only


Open current slide into separate viewport panel

This button opens the current selected slide into a separate viewport tab

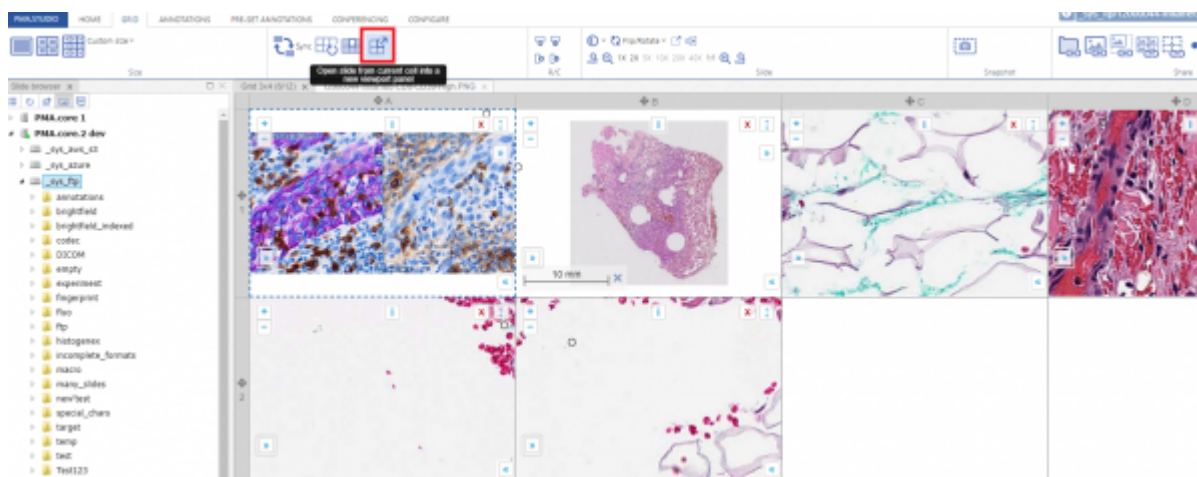

RC


This section allows you to add or remove rows/columns to a grid.

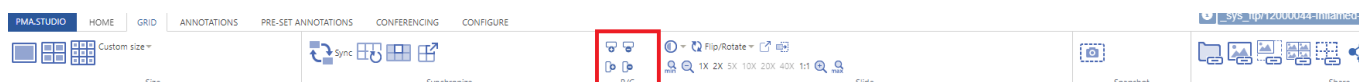

Slide


This section consists of set of tools that includes:

1. Changing contrast/exposure of selected slide
2. Rotation of slide
3. Pop out selected slide
4. Send slide to Tray
5. Zoom in and Zoom out selected slide

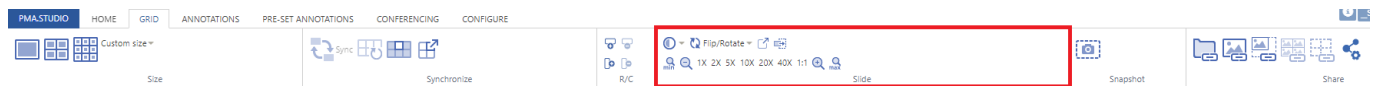


(Changing contrast of selected slide)

This tool allows you to change contrast of selected slide.

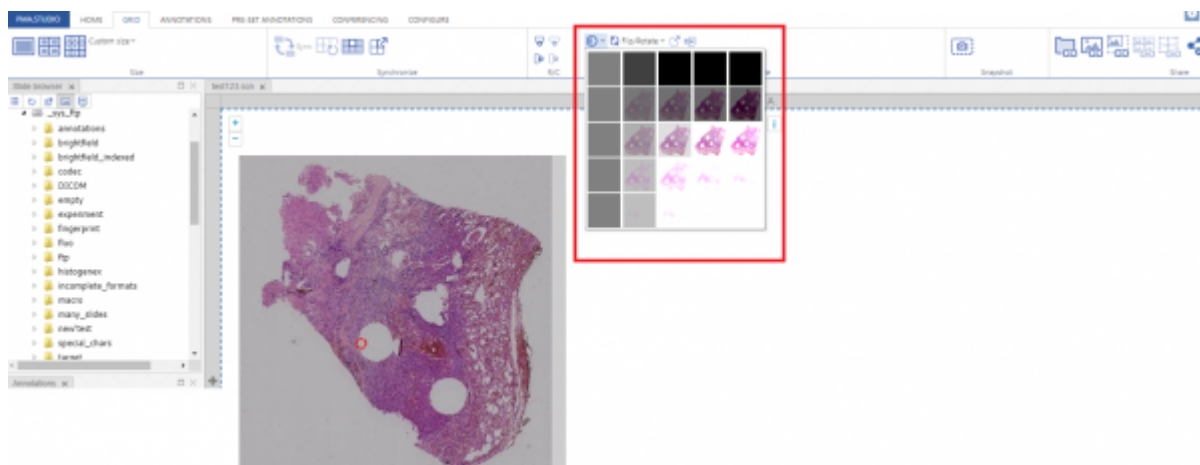


(Rotation of slide)

This tool allows you to rotate slide as per your convenience.

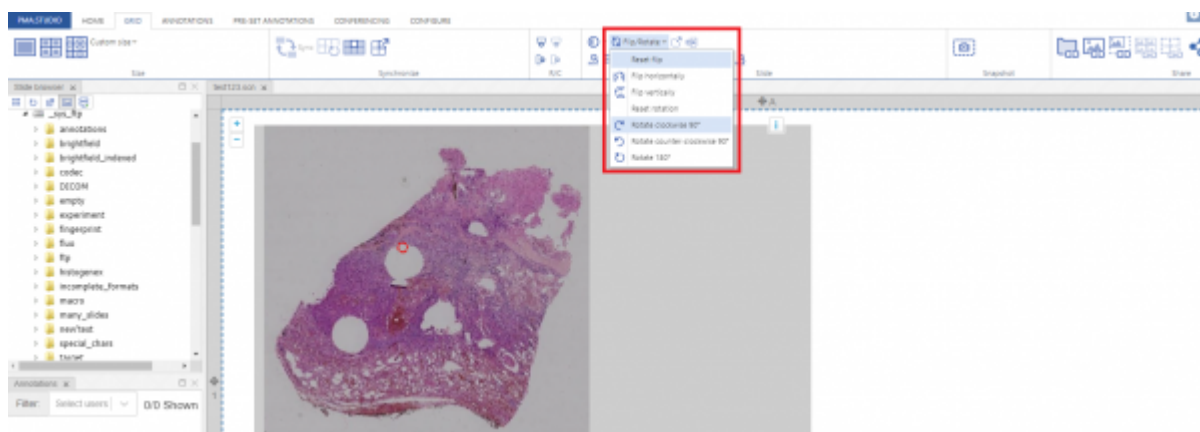


(Pop out selected slide)

Once you click on this tool it generates a shared link for slide and opens it in a new window to view.

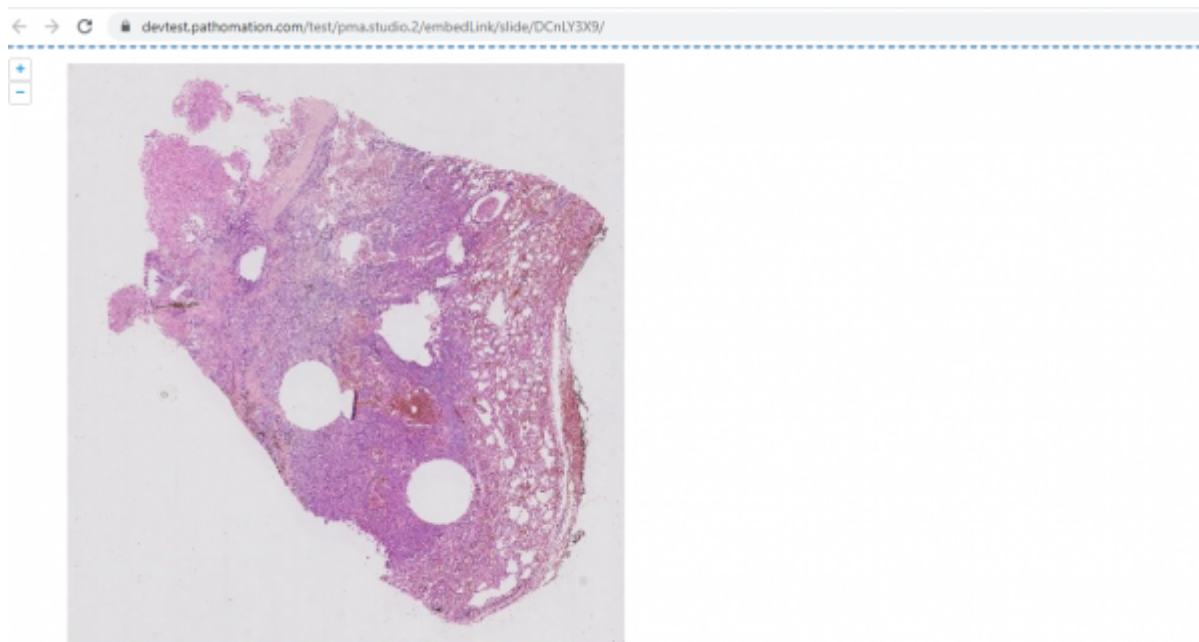


(Send slide to Tray)

This tool sends the selected slide to Tray panel.

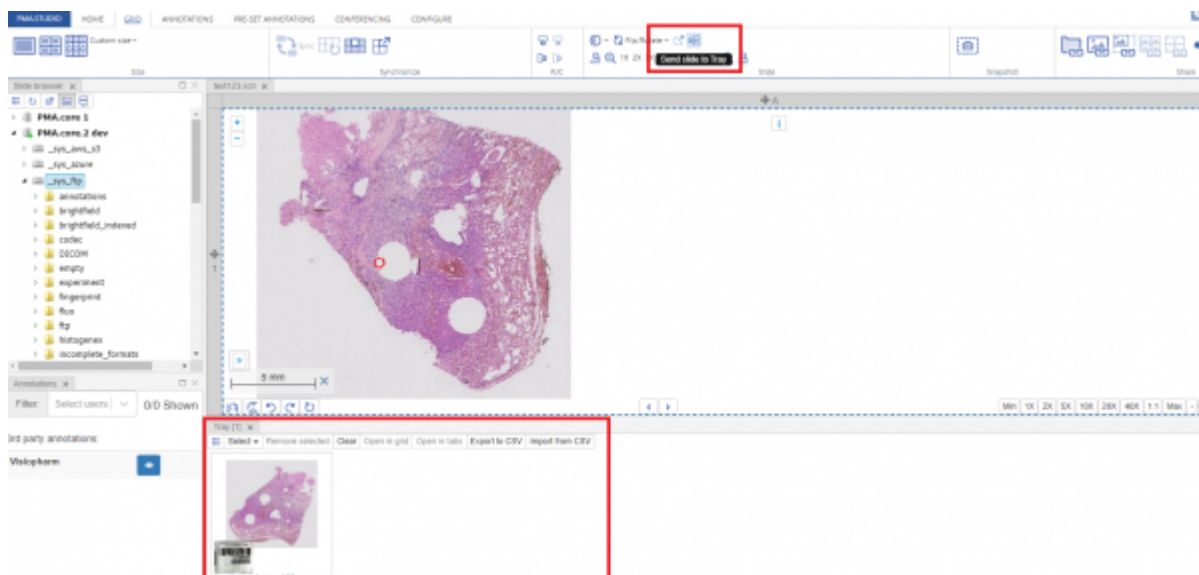

(Zoom in and Zoom out)

PMA.studio provides multiple zooming options to zoom-in and zoom-out selected slide.

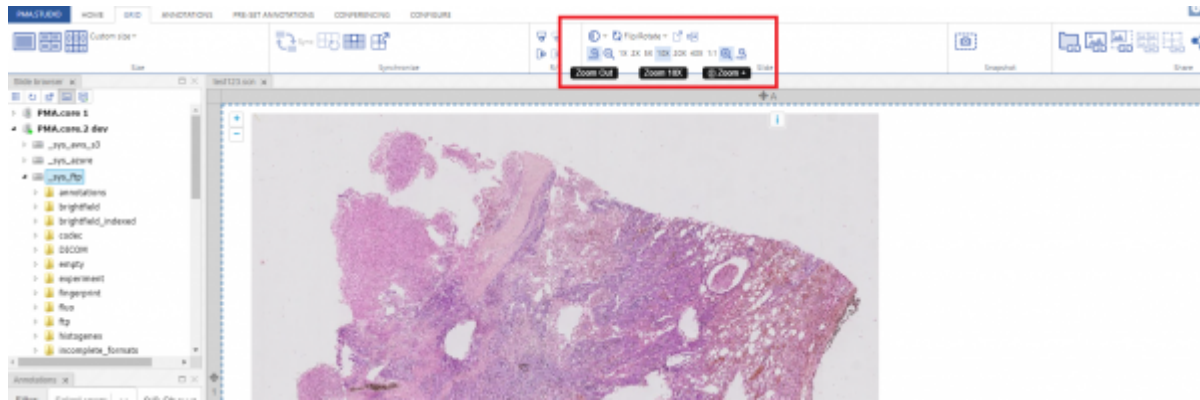

Snapshot

Share

From:
<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:
https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=slides_grid&rev=1674046192

Last update: **2023/01/18 15:49**

