

Sharing Slide(s)


PIMSLS allows you to share content in a variety of ways. There is a dedicated group for sharing content on the [Home](#) and [Grid](#) tabs of the ribbon:



From top left to bottom right the icons are:

1. Quick Share current folder
2. Quick Share current slide
3. Quick Share currently selected ROI / annotation
4. Quick Share current grid
5. Quick Share current ROIs/annotations visible in the grid
6. Quick Share barcode of current slide
7. Quick Share thumbnail of current slide
8. Custom share - this option shows you all the above possibilities and gives you extra customization options

Once you click of any of the Quick Share options you will be shown a popup window that allows you to copy the shared link to clipboard or to scan / download the QR-code

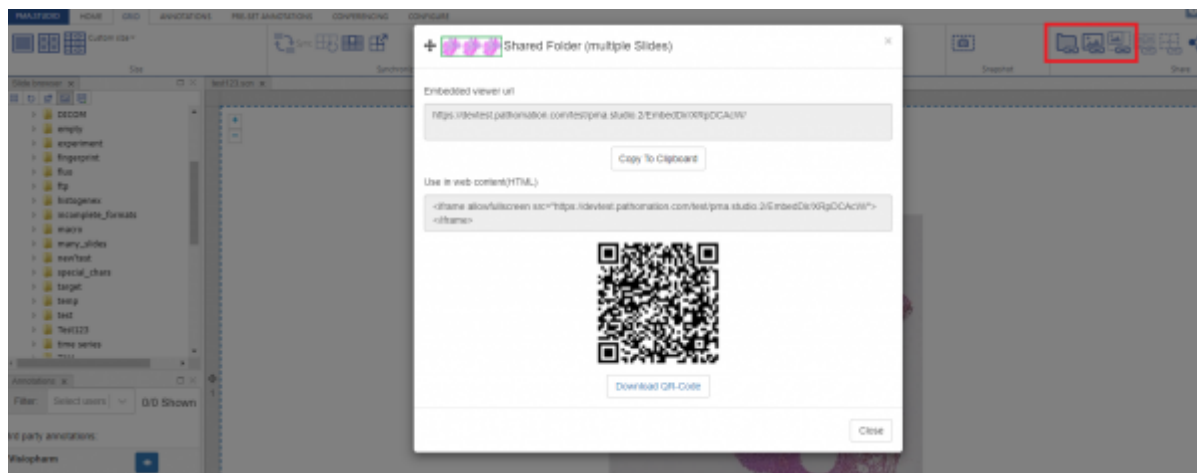


Custom Share

If you want more control over what and how you're sharing content, you can click on the final "Custom Share" button of the group. You could say that that's our "universal" share button.


It allows for further customization of your share link, including:

1. Setting a password to protect your shared content
2. Setting an expiry date (e.g. students can only access it for the duration of a test)
3. Include or exclude the barcode, annotations, and annotation labels from the shared link
4. Give the share a name
5. Add comments

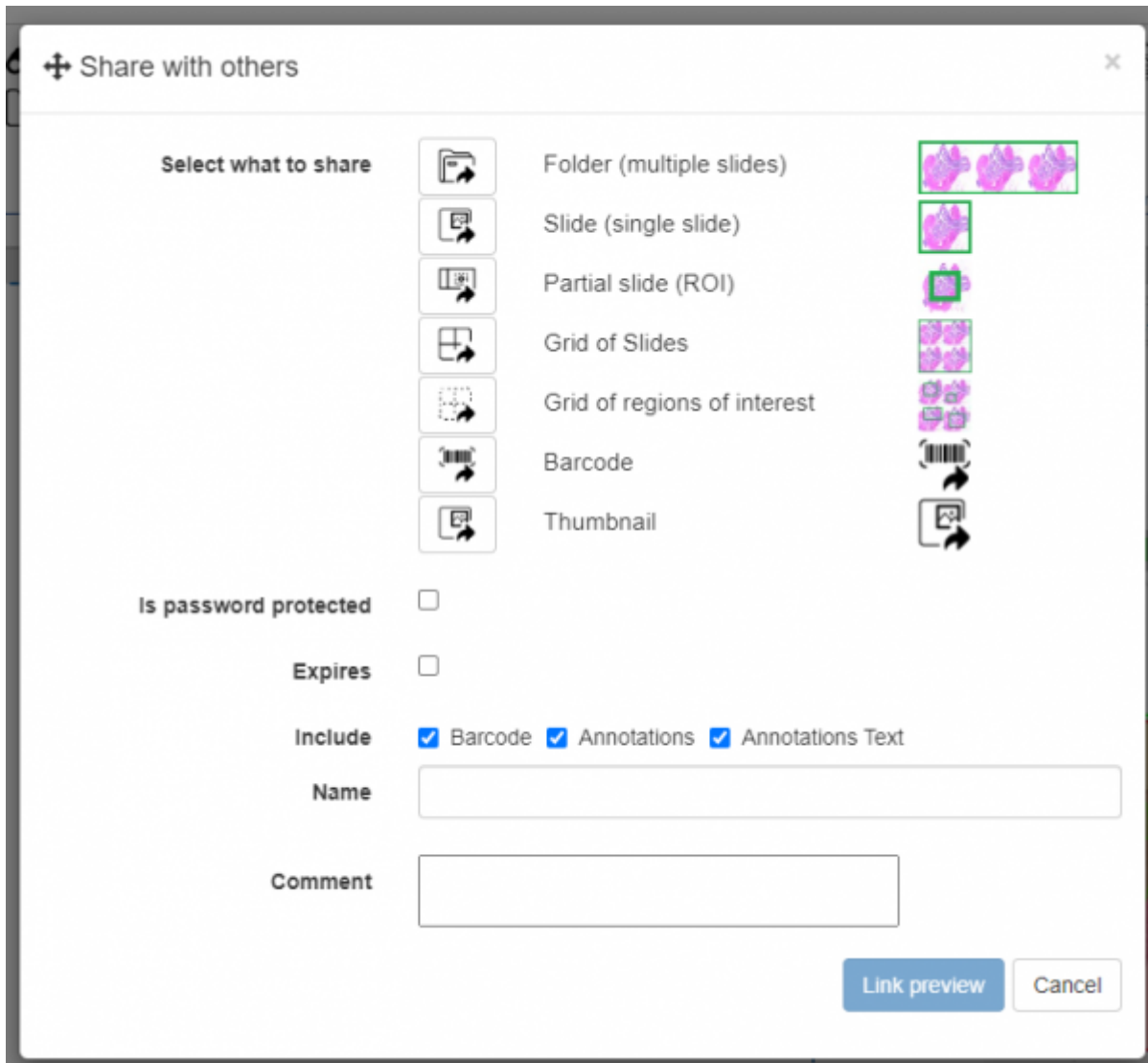


The 'Share with others' dialog box is shown. It has a title bar with a plus icon and a close button. The main content area is divided into two columns. The left column is titled 'Select what to share' and contains seven options, each with an icon: 'Folder (multiple slides)', 'Slide (single slide)', 'Partial slide (ROI)', 'Grid of Slides', 'Grid of regions of interest', 'Barcode', and 'Thumbnail'. The right column shows a preview of the selected 'Folder (multiple slides)' option, displaying a grid of four pink brain scan images. Below the selection options, there are three checkboxes: 'Is password protected' (unchecked), 'Expires' (unchecked), and 'Include' (checked). The 'Include' section has three sub-checkboxes: 'Barcode' (checked), 'Annotations' (checked), and 'Annotations Text' (checked). Below these are two text input fields: 'Name' and 'Comment'. At the bottom right, there are two buttons: 'Link preview' and 'Cancel'.

Password protected preview

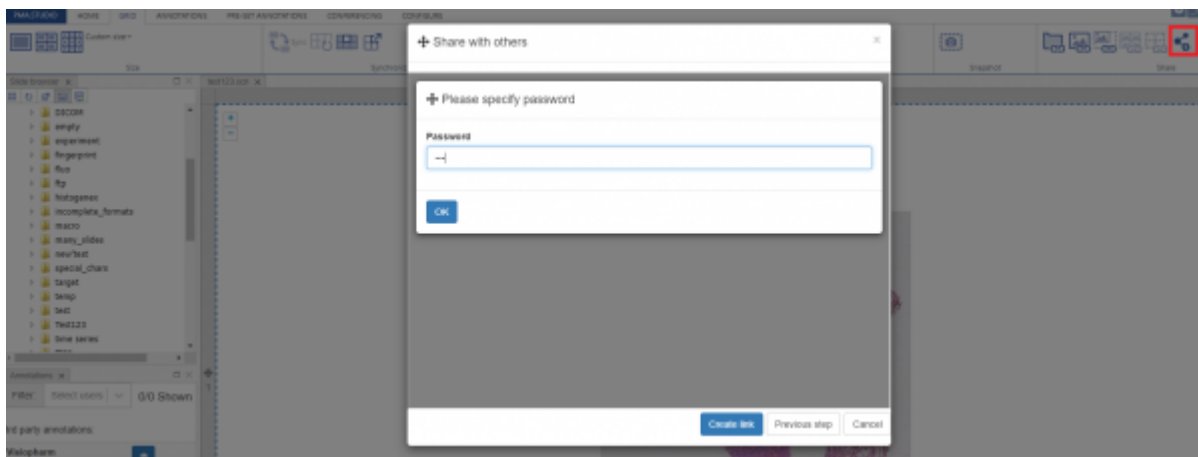


The 'Share with others' dialog box is shown, but it is overlaid by a smaller 'Please specify password' dialog box. The 'Please specify password' dialog box has a title bar with a plus icon and a close button. It contains a text input field labeled 'Password' and a 'OK' button. The 'Share with others' dialog box is partially visible behind it, showing the 'Please specify password' dialog box. The 'Share with others' dialog box is also visible in the background, showing the 'Please specify password' dialog box. The 'Share with others' dialog box is also visible in the background, showing the 'Please specify password' dialog box.

Preview

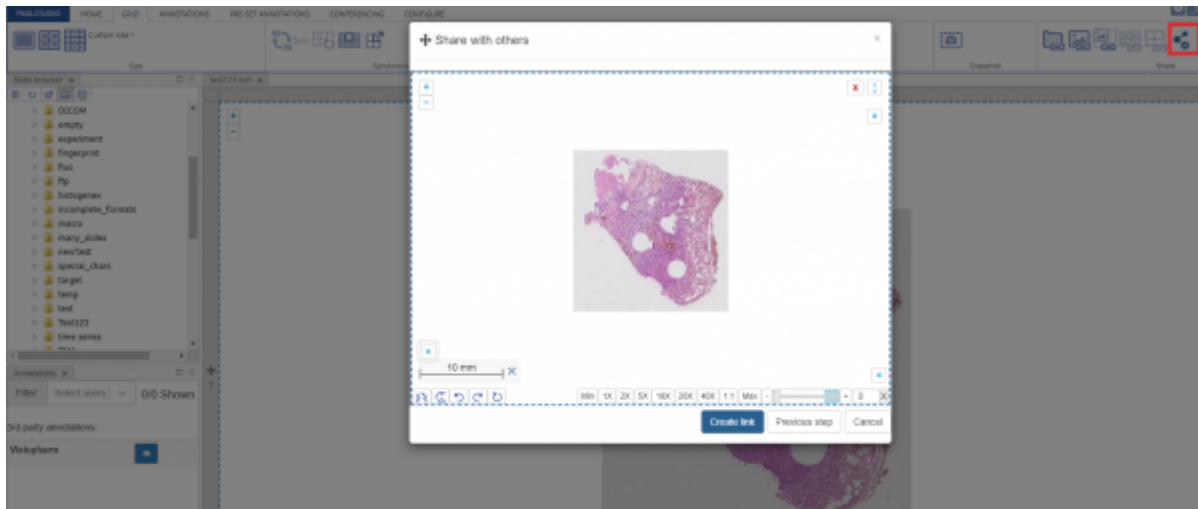

And once you press the “Create Link” button it will generate the share link and show a popup window with option to “Copy to clipboard” or “Download QR-Code”

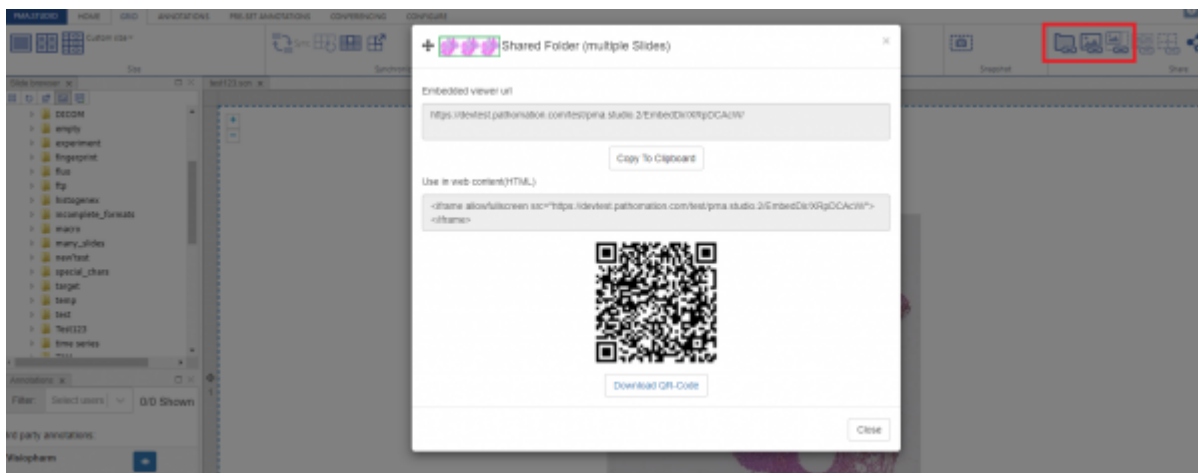

From:
<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:
https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=slides_sharing&rev=1730994071

Last update: **2024/11/07 18:41**

