

Static snapshots

PIMSLS has several snapshot functions on the Home tab.

1. Opens the snapshot menu for the current slide
2. Takes a quick snapshot of the current grid i.e. all open images (also available in the Grid tab)
3. Takes a snapshot of all ROIs / annotations that are currently open and downloads them in a ZIP file



Clicking 2 or 3 will immediately download the results, whereas option 1 will open a popup window. Here you can customize your snap with multiple options related to the slide before saving it. You can specify:

Save snapshot

Select Image Format:

Filename Template:

Scale type: ☒ DPI ☐ Resolution

Show Scale Bar: ☒

Scale bar location:

Scale bar font size:

Scale bar font:

Show Annotations: ☒

Show Slide Overview: ☐

Show Barcode: ☐

Show Slide Label: ☐

Download Archive Of All ROIs From The Grid: ☐

Server: Snapshot Path: _sys_aws_s3/annotations/CMU-1.mnx Image size: 789 * 674 px

Save Snapshot To Clipboard Close

1. Image Format
2. Filename Template
3. Scale type i.e. DPI or resolution based
4. Show Scale Bar - selecting this will prompt options to customize such as font, location
5. Show Annotations
6. Show Slide Overview
7. Show Barcode
8. Show Slide Label
9. Download archive of ROIs

You have two options to save a snapshot, if you press "Save Snapshot" it will generate an image that can be saved on your system or if you press "To Clipboard", it will copy that slide to clipboard and you can simply paste it elsewhere.

From:

<https://docs.pathomation.com/pimsls/3.0.1/> - **PIMS LS 3.0.1**

Permanent link:

https://docs.pathomation.com/pimsls/3.0.1/doku.php?id=slides_snapshot&rev=1731416086

Last update: **2024/11/12 15:54**

