

# Global settings (administration)


This admin page allows us to set global settings for PMA.Studio application. Page is divided into two set of fields:

**Global settings** has 5 fields: ·

1. Language
2. Application IconNo file chosen
3. Email For Help
4. Conference Server Url
5. Snapshot Filename Template

**Annotation settings** has 5 fields: ·

1. Show Annotation Dialog
2. Auto Save Annotations
3. Restart Annotations Drawing
4. Show Slide Info Button
5. Show Full Slide Path In Viewport

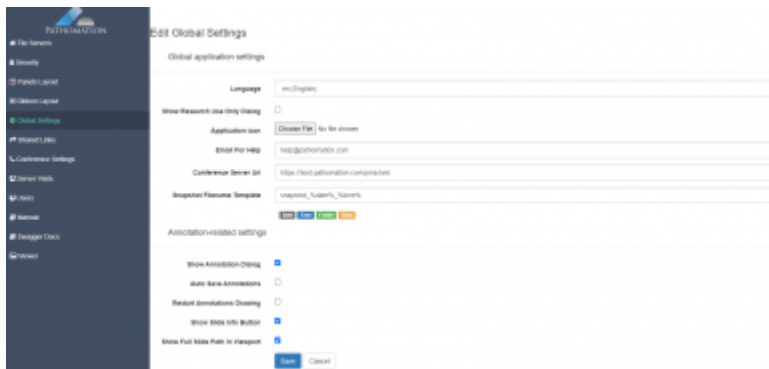

From:  
<https://docs.pathomation.com/pimsls/3.1.0/> - PIMS LS 3.1.0

Permanent link:  
[https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=admin\\_global&rev=1648661029](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=admin_global&rev=1648661029)

Last update: 2022/03/30 20:23

