

Ribbon layout (administration)

This admin page allows you to setup default Ribbon layout that consists of different tabs and controls. Each tab consists of different controls associated to it and you can configure appearance of tabs and controls using this page.

The screenshot shows the 'Edit Ribbon Layout' page in PMA.Studio. The left sidebar is a navigation menu with the following items: Tile Servers, Security, Layout (expanded), Users, Settings (expanded), Logs, Shared Links, Server Visits, Elmah Logs, Docs, and Viewer. Under 'Layout', 'Panels Layout' and 'Ribbon Layout' are visible. The main content area is titled 'Edit Ribbon Layout' and contains a code editor with the following XML configuration:

```

51         icon="sign_out.png"/>
52     </FileTabMenu>
53     <Tab label="Home" hint="Default" name="tab-home" enabled="true" visible="true">
54         <Ribbon label="Layout" width="9%">
55             <Tool type="buttons" size="large">
56                 <Command name="simple-layout" hint="Default" icon="simple_layout.png"/>
57                 <Command name="detailed-layout" hint="with slide info and annotations" icon="detailed_layout.png"/>
58                 <Command name="simple-layout-with-tray" hint="with tray" icon="simple_layout_with_tray.png"/>
59                 <Command name="detailed-layout-with-tray" hint="slide info, annotations and tray" icon="detailed_layout_with_tray.png"/>
60                 <Command name="panels-dialog" hint="More" icon="panels.png" />
61                 <Command name="panels-dialog" props="test" hint="More" icon="panels.png" />
62             </Tool>
63         </Ribbon>
64         <Ribbon label="viewport" width="8%">
65             <Tool type="toggle-buttons" size="small">
66                 <Command name="viewport-toggle-overview" hint="Toggle overview" icon="overview.png"/>
67                 <Command name="viewport-toggle-slide-path" hint="Toggle slide path" icon="slide_path.png"/>
68                 <Command name="viewport-toggle-barcode" hint="Toggle barcode" icon="barcode.png"/>
69                 <Command name="viewport-toggle-rotation-flip" hint="Toggle rotation and flip" icon="rotation_flip.png"/>
70             </Tool>
71             <Tool type="custom-dropdown"
72                 templateId="brightness_contrast_dropdown_template"
73                 name="brightness-contrast-dropdown"
74                 icon="bright_contr.png"/>
75             <Tool type="buttons" size="small">
76                 <Command hint="Create new Viewport panel with 1x1 grid" name="add-tab" icon="add_tab.png"/>
77                 <Command hint="Reset panel layout" name="reset-layout" icon="reset_panels_layout.png"/>
78             </Tool>
79         </Ribbon>
80         <Tool type="break"></Tool>
81         <Tool type="toggle-buttons" size="small">
82             <Command name="viewport-toggle-zoom" hint="Toggle zoom controls" icon="zoom_controls.png"/>
83             <Command name="viewport-toggle-scalebar" hint="Toggle scalebar" icon="scalebar.png"/>
84             <Command name="viewport-toggle-label" hint="Toggle slide label" icon="slide_label.png"/>
85             <Command name="viewport-toggle-layers" hint="Toggle channels control" icon="layers.png"/>
86             <Command name="panel-toggle-color" hint="Toggle color settings panel" icon="panel_toggle_color.png"/>
87         </Tool>
88     </Tab>

```

Below are some of the tabs and corresponding controls used in PMA.Studio.

Home tab has 8 sections:

1. View
2. Grid
3. Snap
4. Magnify
5. Directory
6. Slide
7. Share
8. Search



Grid tab supplies instruments to manage grid and it has 6 sections:

1. ItemSize
2. Synchronize
3. Rows/Columns
4. Slide
5. Snapshot
6. Share



Annotations tab (available in PMA.Studio) supplies instruments for working with annotations and has 4 sections:

1. Annotations
2. Shapes
3. Export
4. Import




Pre-set Annotations tab (available in PMA.Studio) consists of 3 sections:

1. Annotations
2. Stamps
3. Protocols




Conferencing tab has 2 sections:

1. Conferencing
2. Server

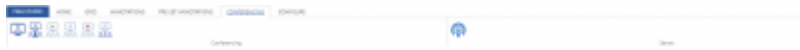

Configure tab has 4 sections:

1. Server
2. Panels
3. Pre-set layouts
4. Viewport

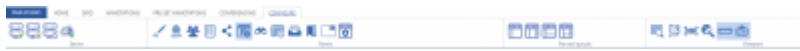

From: <https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link: https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=admin_ribbon&rev=1674470694

Last update: **2023/01/23 13:44**

