

# Edit annotations




Selected annotation


PMA.studio allows you to edit, delete individual, delete all, modify and merge annotations. Once you create annotations they are added to annotation tray panel which allows you to select an annotation and perform certain actions on it.

## Delete


First of the Edit option is to “Delete” an annotation. Once you click the button it shows you a confirmation popup and once you press “Delete” button, annotation gets deleted.

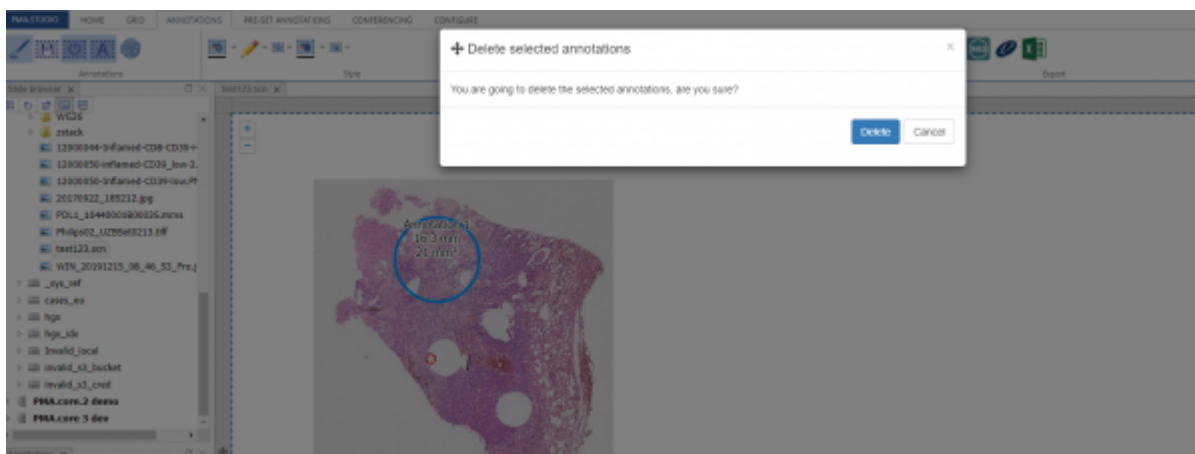

## Delete All Annotations

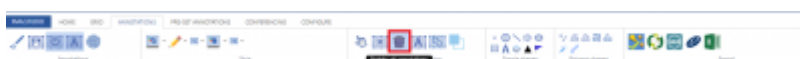


Delete all annotations button allows user to delete all annotations at once. Once you click the button it shows you a confirmation popup and once you press “Delete” button, all annotations get deleted.

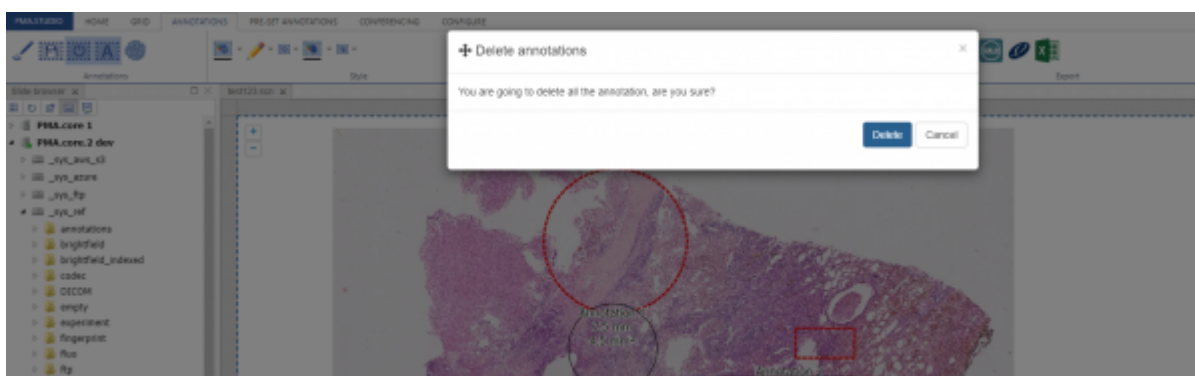

## Edit



“Edit” button allows you to edit an existing annotation. Once you select an annotation from annotation tray panel you see a popup window that allows you to change caption and classification of

annotation. Once you modify the content and press “OK” annotation gets updated.

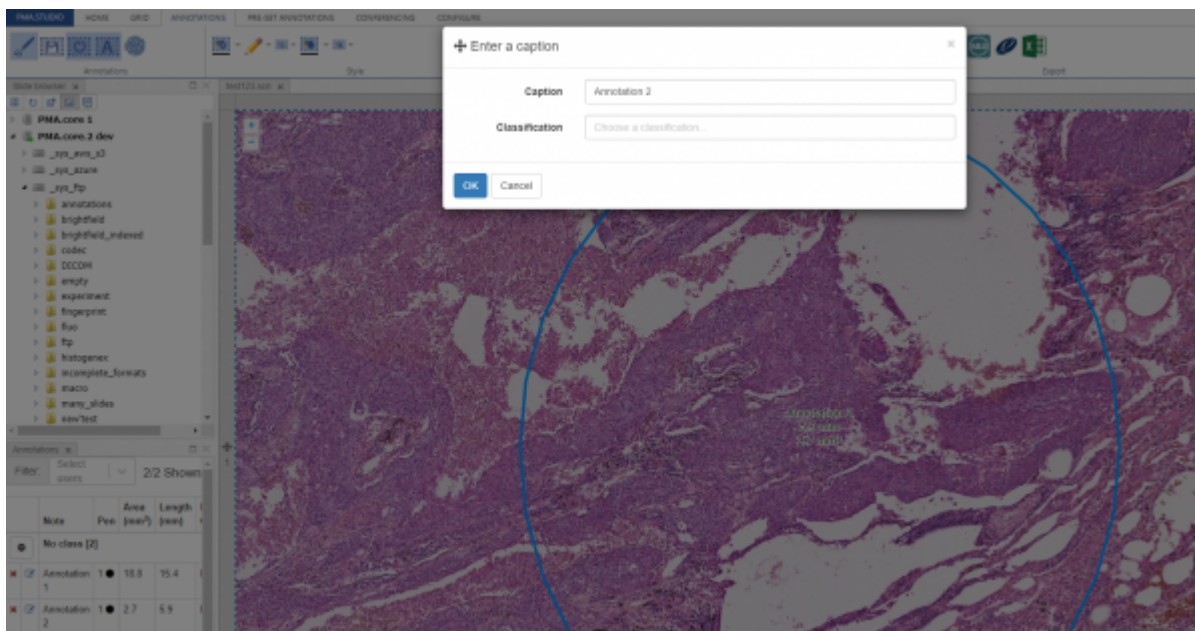


### Merge

Merge selected annotation button allows you to merge two annotations together. Right now PMA.Studio allows merging of “Line” and “Freehand” annotation types. To merge annotations you first select the two annotations using shift key and the press “merge selected” button and both annotations will be merged.

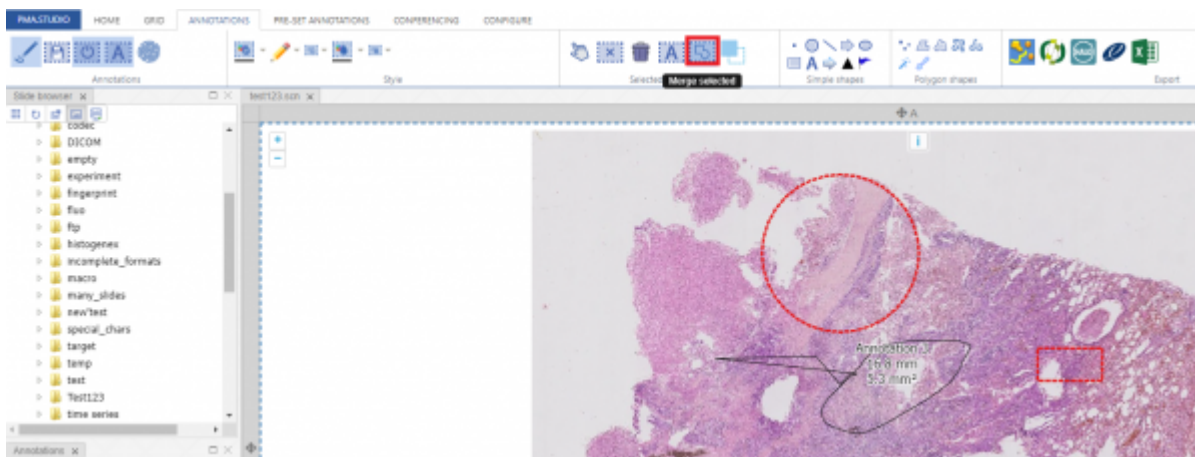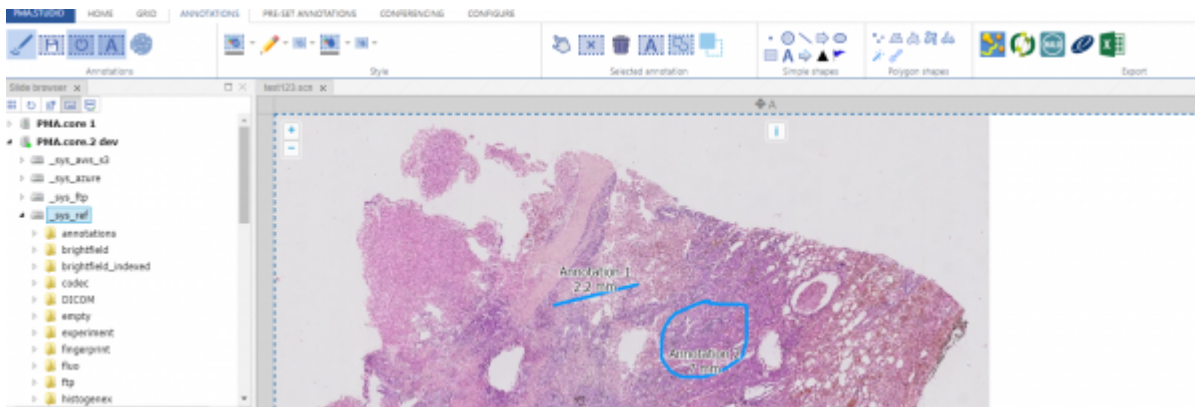

### “Modify”

Modify button allows user to modify an existing annotation. To modify an annotation you need to selected the annotation and then press the “modify” button. Once you are done with the modification, you can save the annotation.

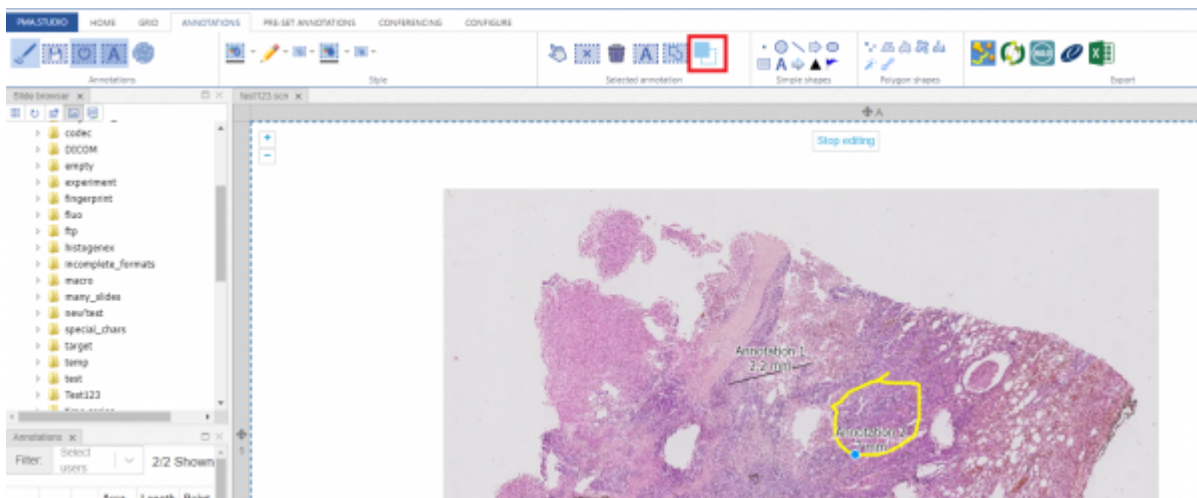

From:  
<https://docs.pathomation.com/pimsls/3.1.0/> - PIMS LS 3.1.0

Permanent link:  
[https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations\\_edit&rev=1658177615](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations_edit&rev=1658177615)

Last update: 2022/07/18 23:53

