


# Annotation Import and Export and 3rd Party Annotations

PIMS LS allows you to import and export annotations in a variety of formats, including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

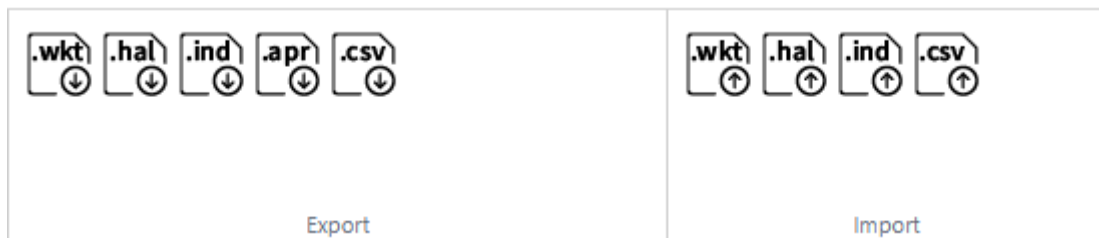


## Export

You can export slide annotations in different formats including Well-known text (the format used by PMA.core internally), Visiopharm mld files, Indica Labs HALO annotations files, Definiens/Aperio xml files, and 'plain' csv files.

## Import

User can also import/upload annotations through PIMS LS. Once you click on the import button for the format you want to import, a menu will appear prompting you to select the file that contains annotation data.

Once a valid file is uploaded annotations are added to the slide.

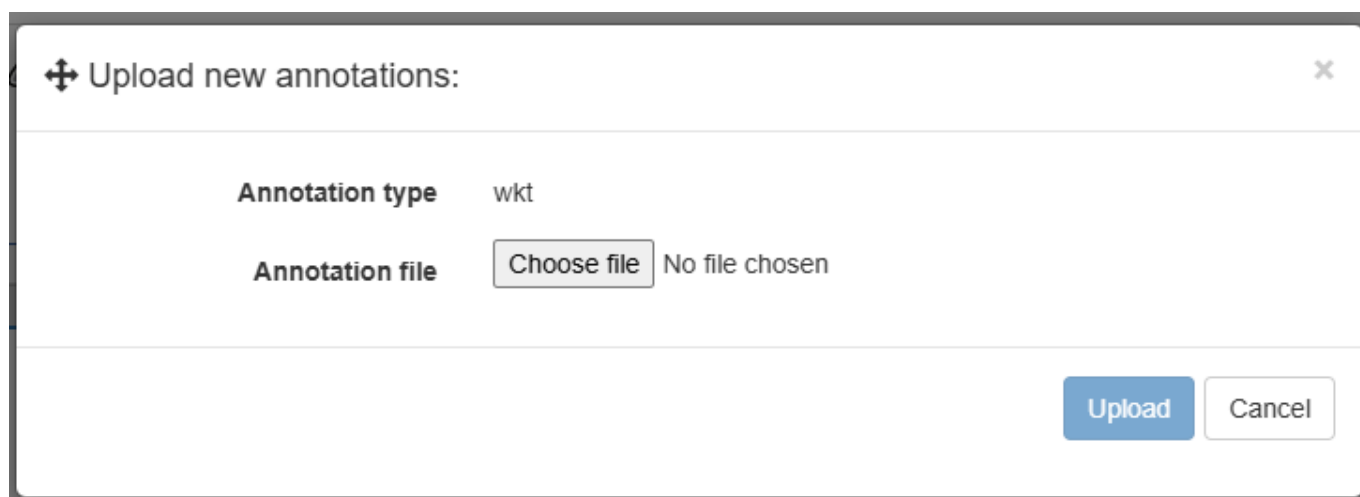

Last update: 2024/11/29 11:38 annotations\_import\_export [https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations\\_import\\_export&rev=1732869525](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations_import_export&rev=1732869525)

---

From: <https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link: [https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations\\_import\\_export&rev=1732869525](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations_import_export&rev=1732869525)

Last update: **2024/11/29 11:38**

