

Brush and magic wand


Brush and magic wand are two special tools that allow you to draw annotation on specific area of a slide.

(Brush Tool)

Brush tool allows you to paint specific area and create annotation on it.

(Wand Tool)

Wand tool allows you to select different areas on a slide by simply clicking on the slide. Once you click on a slide that specific area is automatically selected and annotation can be created on that area by clicking "create annotation from selection" button.

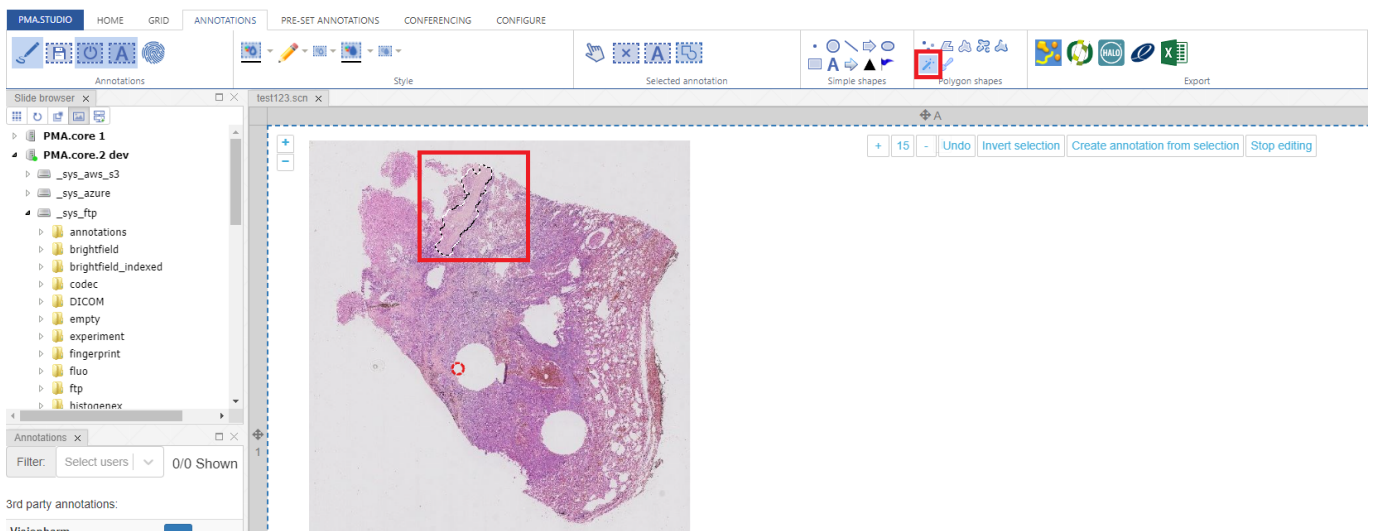

From: <https://docs.pathomation.com/pimsls/3.1.0/> - PIMS LS 3.1.0

Permanent link: https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations_magic&rev=1650563054

Last update: 2022/04/21 20:44

