


# Simple shapes


Below are the set of simple shapes that can be added to a annotation.

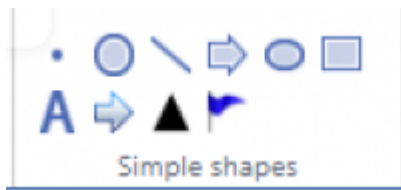

- Point
- Circle
- Line
- Arrow
- Ellipse
- Rectangle
- Text
- Landmark 1
- Landmark 2
- Landmark 3


## Point Shape

Lets start with drawing a point, once you select the shape and click on the slide; a point will be drawn and a popup dialog would appear that would allow you to add annotation title and classification. Once you press "Add Annotation" the annotation will be saved and will appear on the slide.

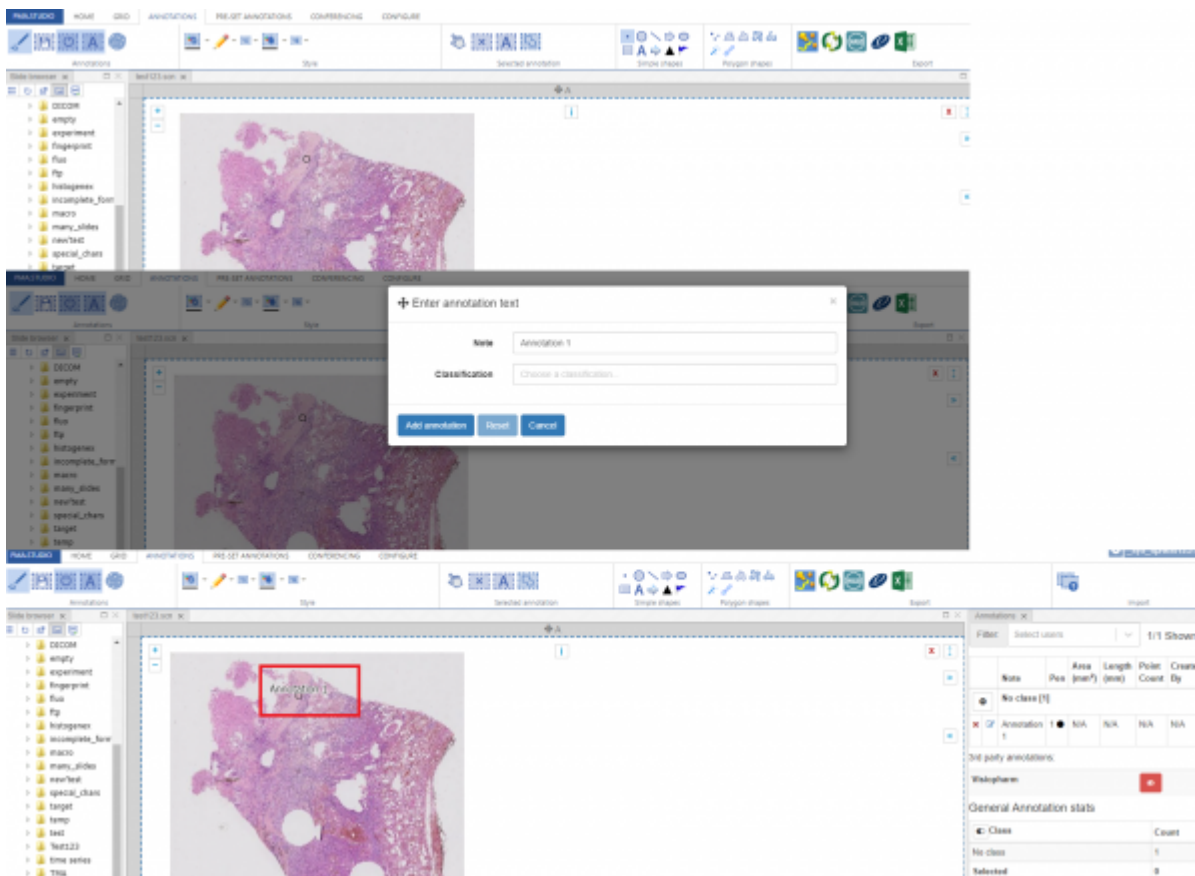


## Circle Shape

Next is “Circle” tool that allows you to draw circle annotation on a slide.

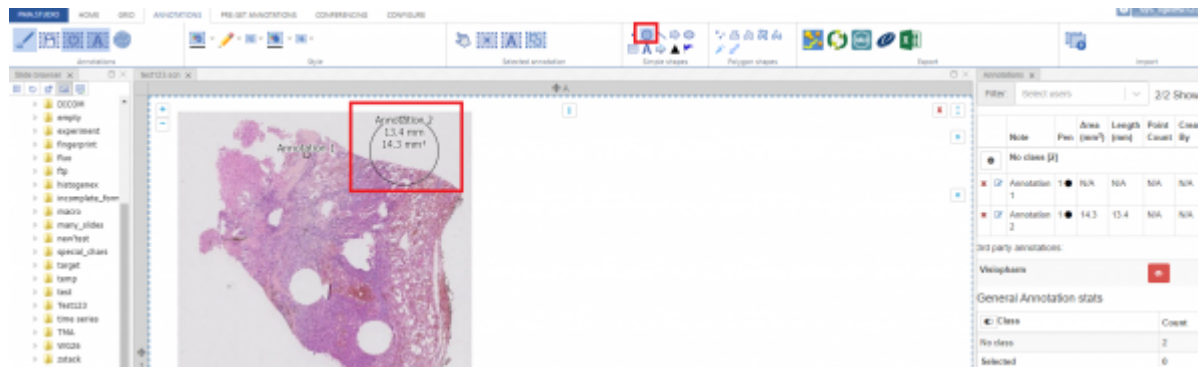


## Line Tool

Next is “Line” tool that allows you to draw line annotation on a slide.

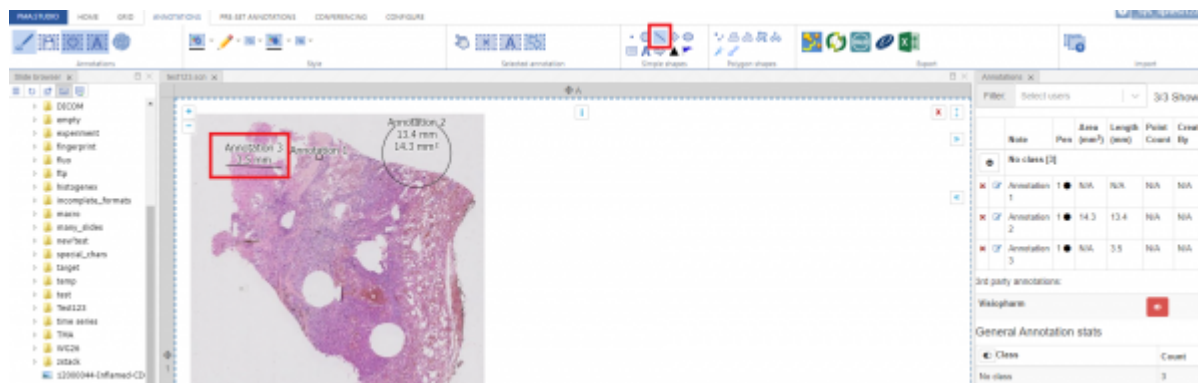


## Arrow Tool

Next is “Arrow” tool that allows you to draw arrow annotation on a slide.

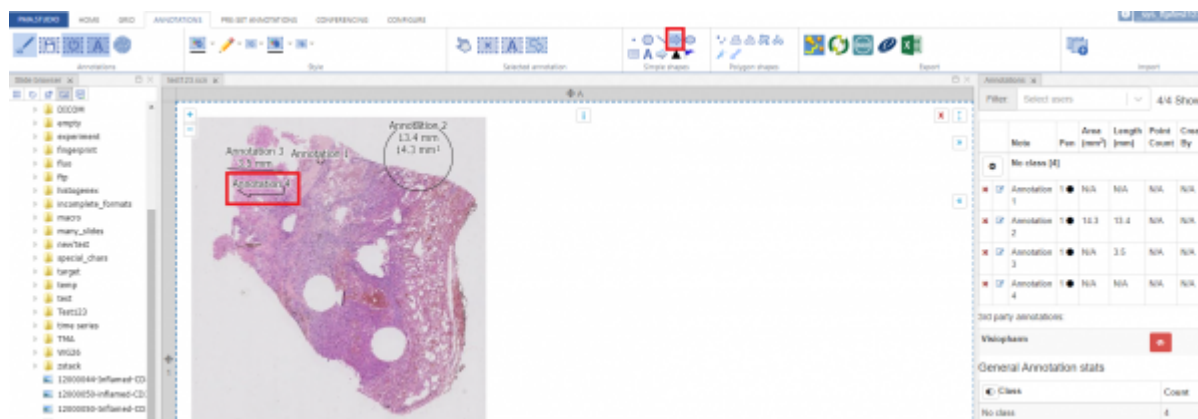

## Ellipse Tool

Next is "Ellipse" tool that allows you to draw ellipse annotation on a slide.

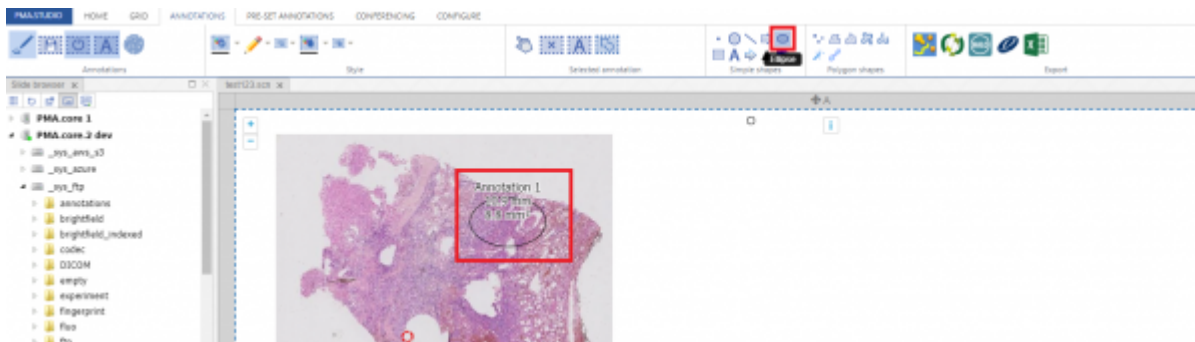


## Rectangular Tool

Next is "Rectangular" tool that allows you to draw rectangular annotation on a slide.



## Landmark 1 Tool

Next is "Landmark 1" tool that allows you to draw landmark annotation on a slide.

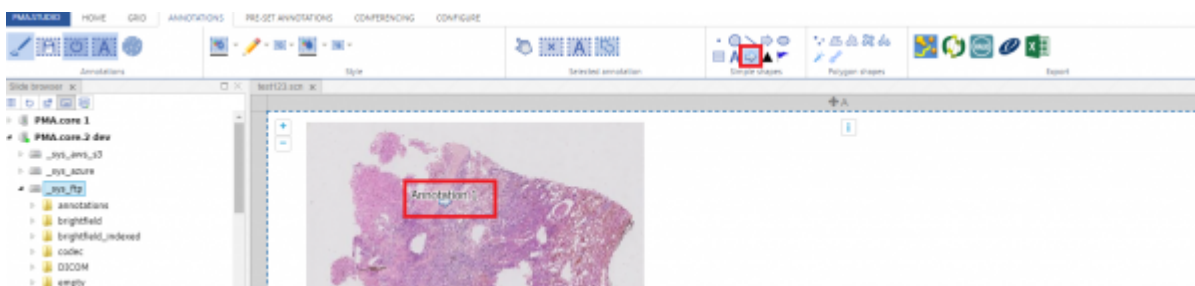

From: <https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link: [https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations\\_simple&rev=1650554900](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=annotations_simple&rev=1650554900)

Last update: **2022/04/21 18:28**

