


Configure Server

The Server toolgroup in the configure tab contains everything you need to connect to different kinds of storage.

Not only can you connect to other PMA.Cores, but also to storage on Pathomation's cloud platform [My Pathomation](#) if you have an account - FYI your first 10 GB is free!

You can also connect to [PMA.start](#), our desktop application for local storage connection.

Connecting to another PMA.core

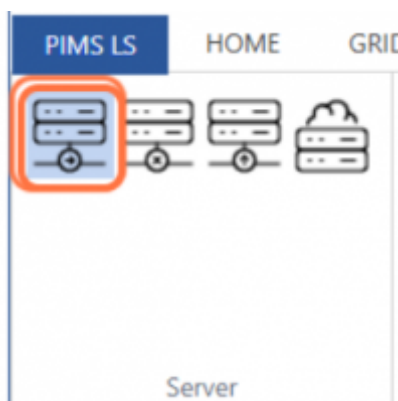

Clicking the first icon will open the menu for connecting to another PMA.core, pictured below:

A screenshot of a dialog box titled 'Quick connect to Tile server'. The dialog has a close button (X) in the top right corner. It contains the following fields and controls:

- Server Name**: A text input field with the placeholder text 'Server Name'.
- Server Url**: A text input field with the placeholder text 'Tile Server Url'.
- Username**: A text input field with the placeholder text 'Username' and a red eye icon to toggle visibility.
- Password**: A text input field with the placeholder text 'Password' and a red eye icon to toggle visibility.
- Save**: A checkbox with a label 'Save' and an information icon.
- Check Connection**: A button.
- Add Server**: A blue button.
- Reset**: A blue button.
- Close**: A button in the bottom right corner.

Here you can input:

1. Server name: this is a nickname for the Core/Drive you want to connect to
2. Server URL: this is the web address of the PMA.core you want to connect to e.g. `mycore.mybusiness.com/core/`


3. Username: Your username for logging into the PMA.core
4. Password: Your password for logging into the PMA.core

You can check the connection to see if all the details work.


If you need to be connected to a VPN to access your PMA.core, ensure that you are connected to the right VPN.

Disconnect Server

Clicking this button will disconnect you from all PMA.core servers you have manually connected. Please note that any servers that are set to be automatically logged into will not be affected.

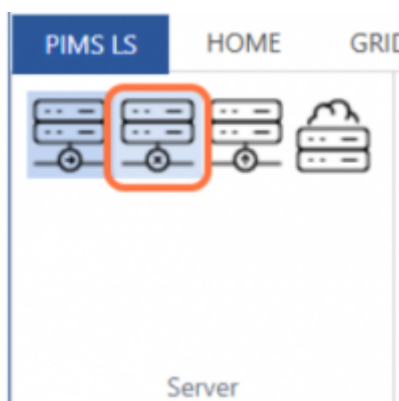


Connect to PMA.start

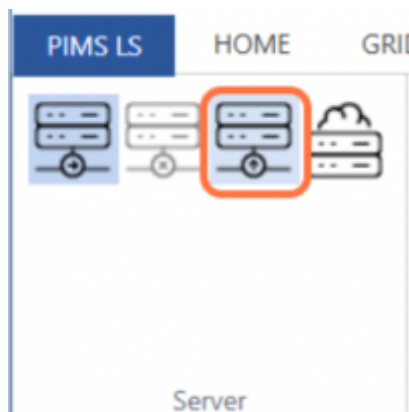


The third icon will search for an active instance of PMA.start on your device and integrate it in your slide browser


Please note - ensure that PMA.start is running before clicking the button

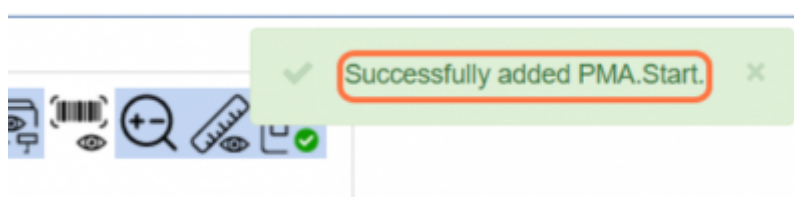

Connect to My Pathomation

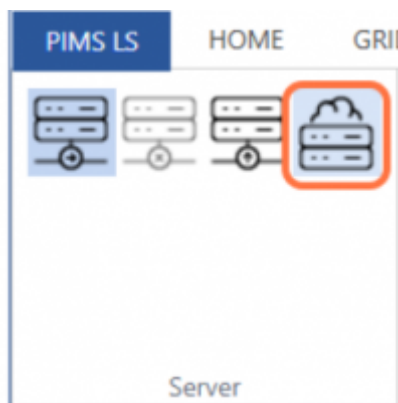

You can also connect to your MyPathomation cloud account
Simply login with your email address and password as you would on my.pathomation.com and your content will seamlessly be added.

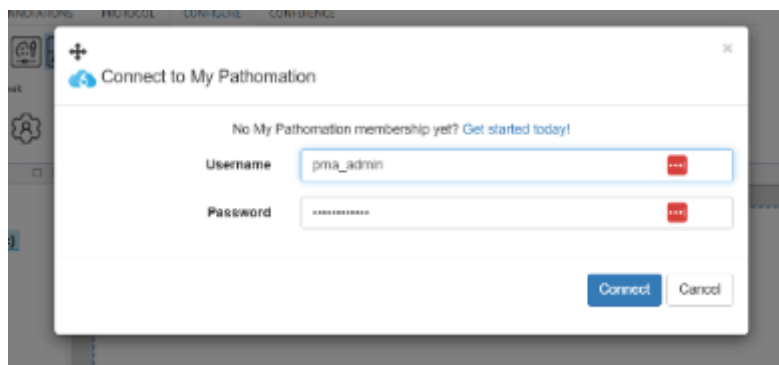

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=configserver>

Last update: **2024/12/04 16:01**

