


Home: Folder tools group


The Folder Tools group in the Home tab contains controls for searching & navigating folders

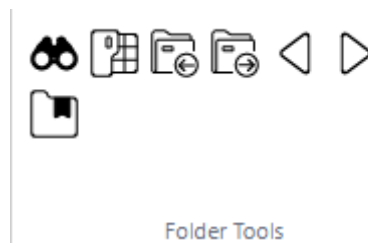

The icons, from left to right

Top Row

- 1. - Search icon - [read more about searching here](#)
- 2. - Open current folder in grid - opens the menu below:

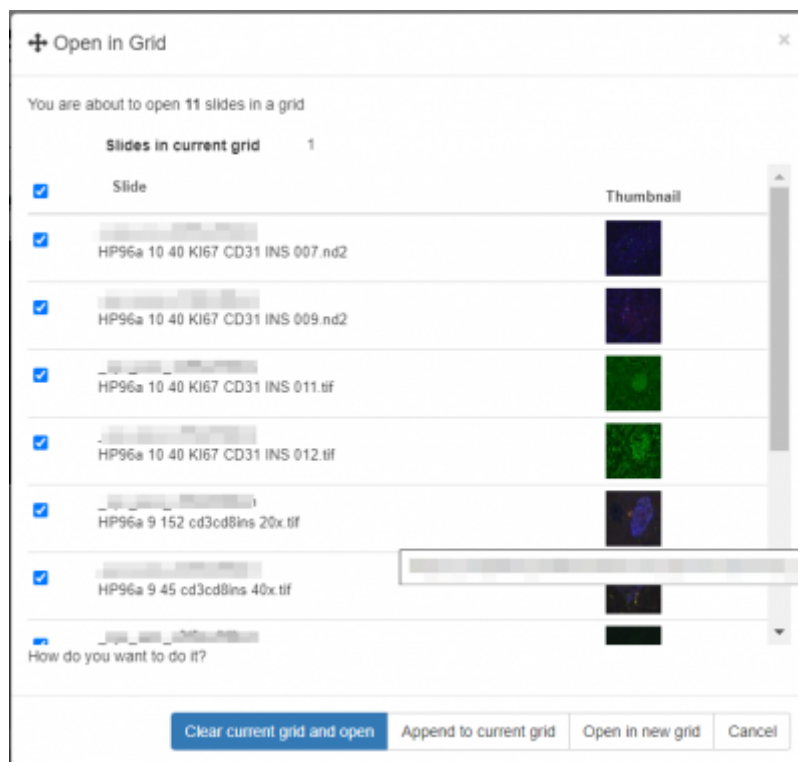

Here you can choose images from the currently highlighted folder in the slide browser to open into your current grid, or make a new grid.

- 3. - Previous folder - selects and opens the previous folder
- 4. - Next folder - selects and opens the next folder
- 5. - Previous file - opens previous image in folder
- 6. - Next file - opens next image in folder

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=folder_tools&rev=1731586974

Last update: **2024/11/14 15:22**

