


Home: Grid Tool

The Grid tool contains icons for synchronising different viewports in the grid, clearing the grid, and opening new grid tabs.

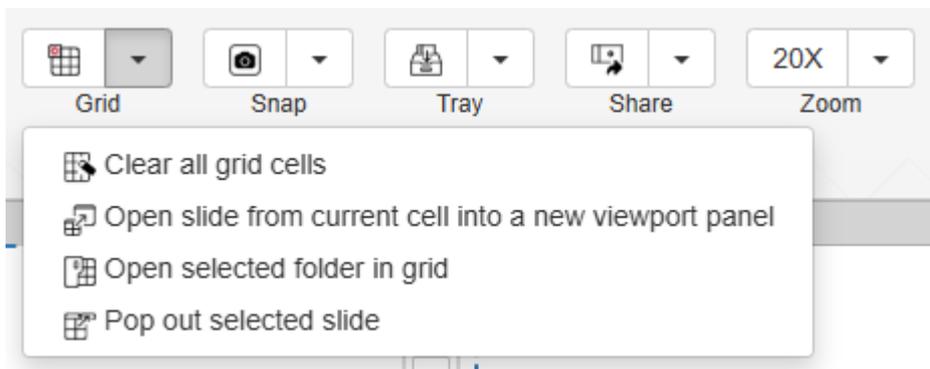

- Clear all grid cells - clears all images from current grid
- Opens current folder in grid - opens the menu below

Open in Grid



You are about to open 4 slides in a grid

Slides in current grid 1

| <input checked="" type="checkbox"/> | Slide | Thumbnail |
|-------------------------------------|--|-----------|
| <input checked="" type="checkbox"/> | _sys_ref/annotations CMU-1.mrxs | |
| <input checked="" type="checkbox"/> | _sys_ref/annotations CMU-1.ndpi | |
| <input checked="" type="checkbox"/> | _sys_ref/annotations CMU-1.svs | |
| <input checked="" type="checkbox"/> | _sys_ref/annotations CMU-1-JP2K-33005.svs | |

How do you want to do it?

- Opens slide from current cell into a new viewport panel
- Pops out selected slide in a new window.
- Clear current cell - clears image from currently selected grid cell

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=grid&rev=1746901236>

Last update: **2025/05/10 21:20**

