


Home: Grid Tool

The Grid tool contains icons for synchronising different viewports in the grid, clearing the grid, and opening new grid tabs.

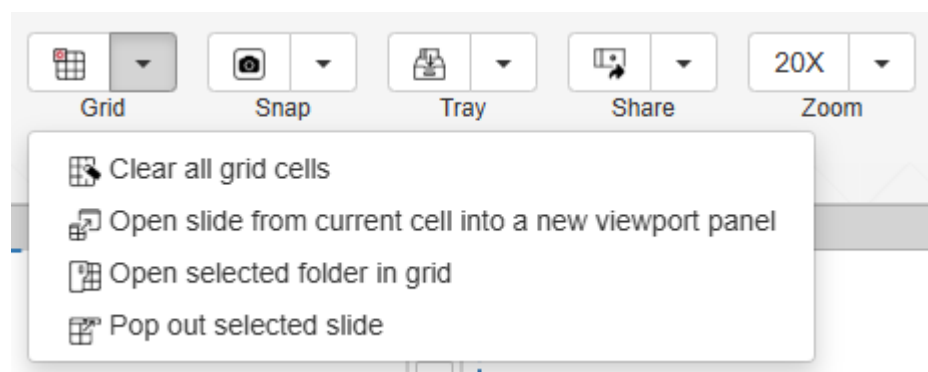

- Clear all grid cells - clears all images from current grid
- Opens current folder in grid - opens the menu below

✚ Open in Grid



You are about to open **4** slides in a grid

Slides in current grid 1

<input checked="" type="checkbox"/>	Slide	Thumbnail
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1.mrxs</code>	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1.ndpi</code>	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1.svs</code>	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations CMU-1-JP2K-33005.svs</code>	

How do you want to do it?

Clear current grid and open

Append to current grid

Open in new grid

Cancel

- Opens slide from current cell into a new viewport panel
- Pops out selected slide in a new window.
- Clear current cell - clears image from currently selected grid cell

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=grid&rev=1746901236>

Last update: **2025/05/10 21:20**

