


# Home: Grid Tool

The Grid dropdown contains tools for synchronizing different viewports in the grid, clearing the grid, and opening new grid tabs.

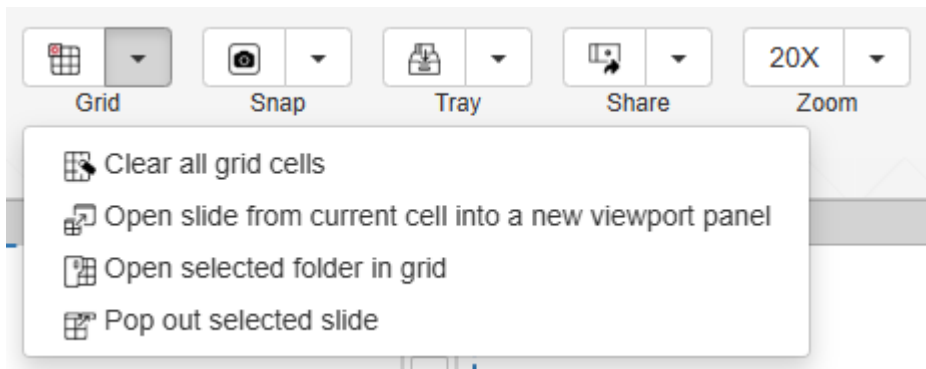

- Clear all grid cells - clears all images from current grid
- Opens current folder in grid - opens the menu below

**+** Open in Grid ✕

You are about to open 4 slides in a grid

**Slides in current grid** 1

<input checked="" type="checkbox"/>	Slide	Thumbnail
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations</code> CMU-1.mrxs	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations</code> CMU-1.ndpi	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations</code> CMU-1.svs	
<input checked="" type="checkbox"/>	<code>_sys_ref/annotations</code> CMU-1-JP2K-33005.svs	

How do you want to do it?

- Opens slide from current cell into a new viewport panel
- Pops out selected slide in a new window.
- Clear current cell - clears image from currently selected grid cell

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=grid&rev=1747058832>

Last update: **2025/05/12 17:07**

