


# Home: Grid Tool

The Grid dropdown contains tools for clearing the grid, opening new grid tabs, open current folder in grid and popping out selected slide in new window.

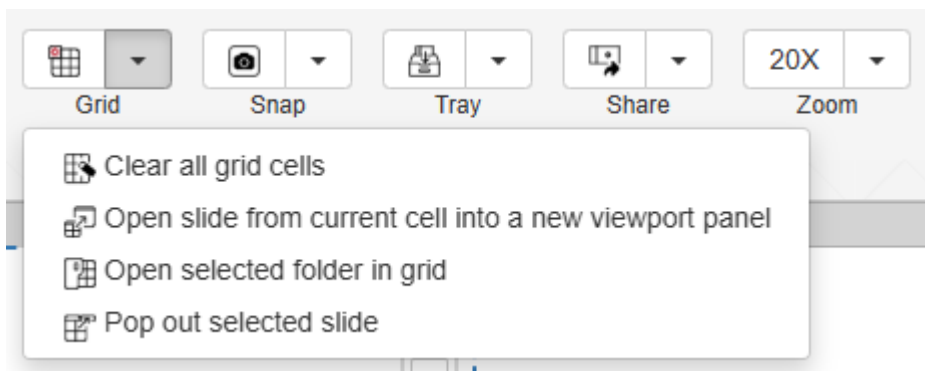

- Clear all grid cells - clears all images from current grid
- Opens current folder in grid - opens the menu below

## Open in Grid



You are about to open 4 slides in a grid

Slides in current grid 1

<input checked="" type="checkbox"/>	Slide	Thumbnail
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1.mrxs	
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1.ndpi	
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1.svs	
<input checked="" type="checkbox"/>	_sys_ref/annotations CMU-1-JP2K-33005.svs	

How do you want to do it?

- Opens slide from current cell into a new viewport panel
- Pops out selected slide in a new window.
- Clear current cell - clears image from currently selected grid cell

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=grid&rev=1747058912>

Last update: **2025/05/12 17:08**

