


# The Grid Tab

The Grid tab is the second tab in the PIMS LS ribbon.

Here you will find tools to create grids of images i.e. view more than one image at a time. It's a very flexible system, allowing you to view anywhere from 2 to 36 images simultaneously - provided your screen is big enough!

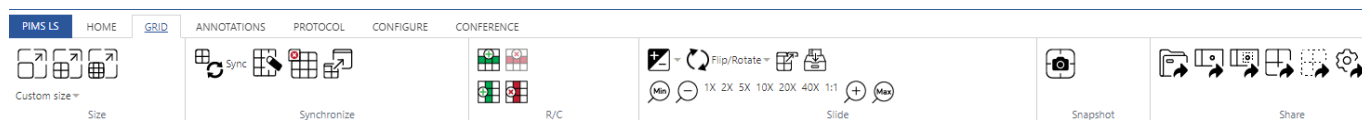

There are dedicated sub articles on the [Size](#), [Synchronize](#) and [Slide](#) tool groups.

The other sections are:

- The R/C toolgroup, which contains buttons to add or remove a rows and columns to the grid
- The snapshot button [whose functions are already covered in the Home section](#)
- The share toolgroup, [whose functions are already covered in the Home section](#)

From:  
<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:  
<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=gridtab&rev=1731940082>

Last update: **2024/11/18 17:28**

