


# The Grid Tab

The Grid tab is the second tab in the PIMS LS ribbon.

Here you will find tools to create grids of images i.e. view more than one image at a time. It's a very flexible system, allowing you to view anywhere from 2 to 36 images simultaneously - provided your screen is big enough!

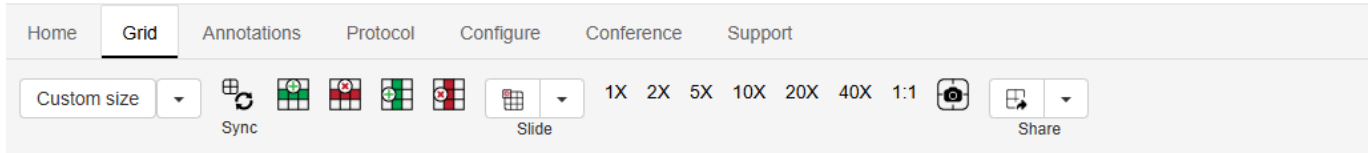

There are dedicated sub articles on the [Size](#), [Synchronize](#) and [Slide](#) tool groups.

The other sections are:

- Sync - [click for more details](#)
- Add row - Adds a new row to the grid on viewport
- Remove row - removes selected row from grid
- Add column - Adds new column to the grid
- Remove column - Removes selected column from the grid

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=gridtab&rev=1747062213>

Last update: **2025/05/12 18:03**

