

# Conferencing panel

Conferencing is fantastic for collaborative second reviews and demonstrations to colleagues and students.

With it you can have the full multi-head microscope from anywhere in the world.

Just head to the conference panel at the top of the screen, click the start button, and share the link with your audience.

Check out what it can look like in the video below:

[pims\\_ls\\_conferencing.mp4](#)

## Setting up a conference

You can either head to the conference tab, or open the conference panel.

Conferences can be in different states:

1. No active conference: when there is no active conference this panel shows a placeholder that allows user to Create new conference or Join an existing conference.
2. Active conference. When you've started a conference you will see:
  1. Link to share - 'Copy' button allows to copy link to clipboard, and then this link can be used for joining a conference as participant
  2. Owner - nickname of user who created a conference
  3. Nickname - nickname of current user
  4. Live mode - can be in 2 states - enabled (green circle), and disabled (red circle). When live mode is enabled conference participants can see the same ROIs as admin, when it is disabled participant cannot see
  5. Everyone in control - indicator is the same as for Live mode, when everyone in control enabled all participants can control what is being shown in conference
  6. Participants - list of participant nicknames
3. Active conference, user is participant: When user joined to a conference as participant (s)he can see a reduced list of fields that are the same to one that is described in previous list item

P.S The conferencing feature of PIMSLS works in close collaboration with [PMA.live](#). A [PMA.live server component must be configured explicitly in the PIMSLS administrative back-end](#) in order for this functionality to work.

From:  
<https://docs.pathomation.com/pims/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:  
[https://docs.pathomation.com/pims/3.1.0/doku.php?id=panel\\_conference&rev=1732893267](https://docs.pathomation.com/pims/3.1.0/doku.php?id=panel_conference&rev=1732893267)

Last update: **2024/11/29 18:14**

