


# Forms panel


[pimsls\\_formshowto.mp4](#)

The “Forms data” panel purpose is to show filled forms for the slide and provide access to editing and filling new forms for currently selected slide.


The panel list downs existing forms and also provide option to add/edit forms. To add a form user can click on the “Add” link on the right hand corner which will popup dialog box that shows up list of forms to choose from. Once user selects a form and press “add” button it loads the respective form to the panel.

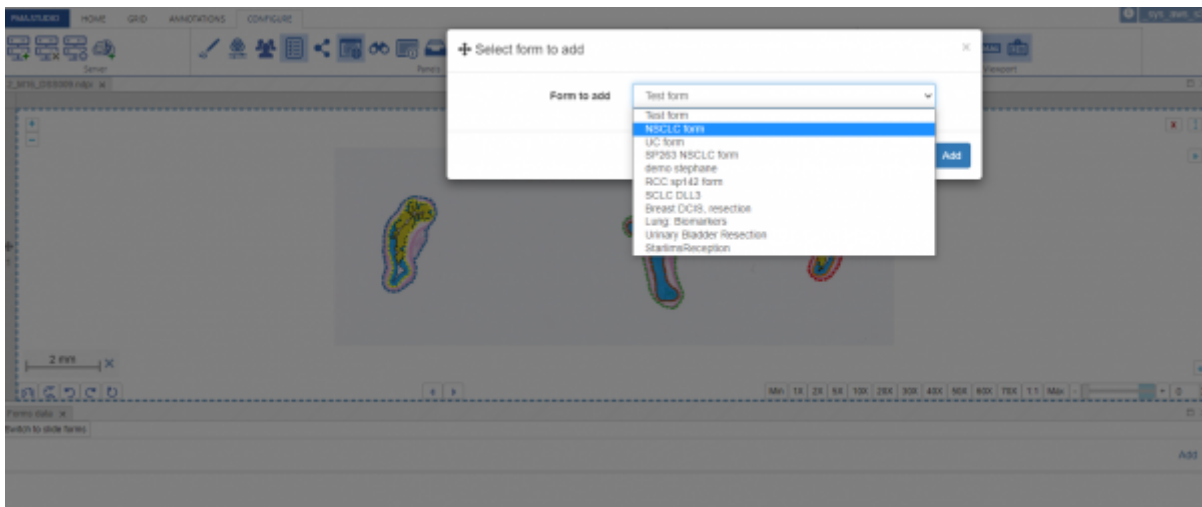

The selected form appears in a new tab that would be filled by the user. Once filled user can save the details and later can edit as well.

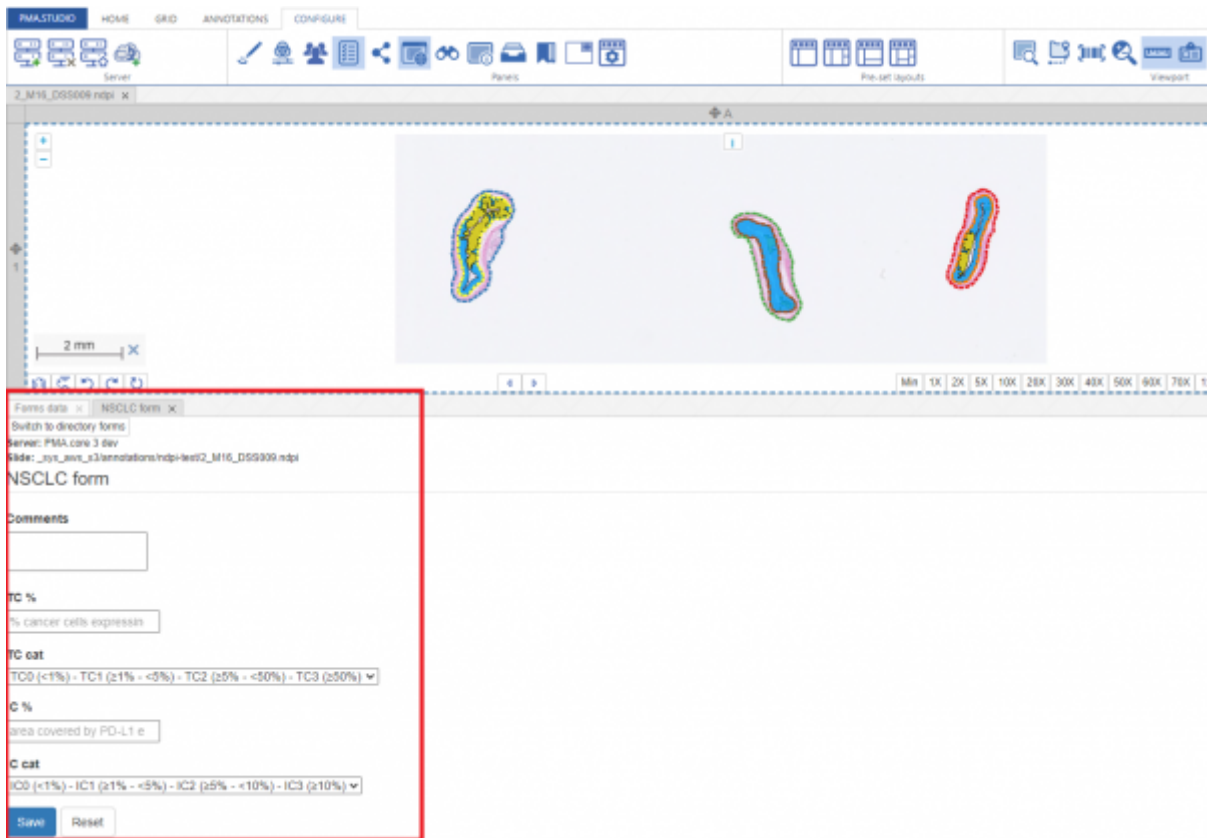

The saved form appears in the list of forms already created/saved. The forms appear as toggle panels where user can expand/collapse a selected form and it will load details of the form user submitted earlier. The list display shows form name and also the user who created that form against selected slide.

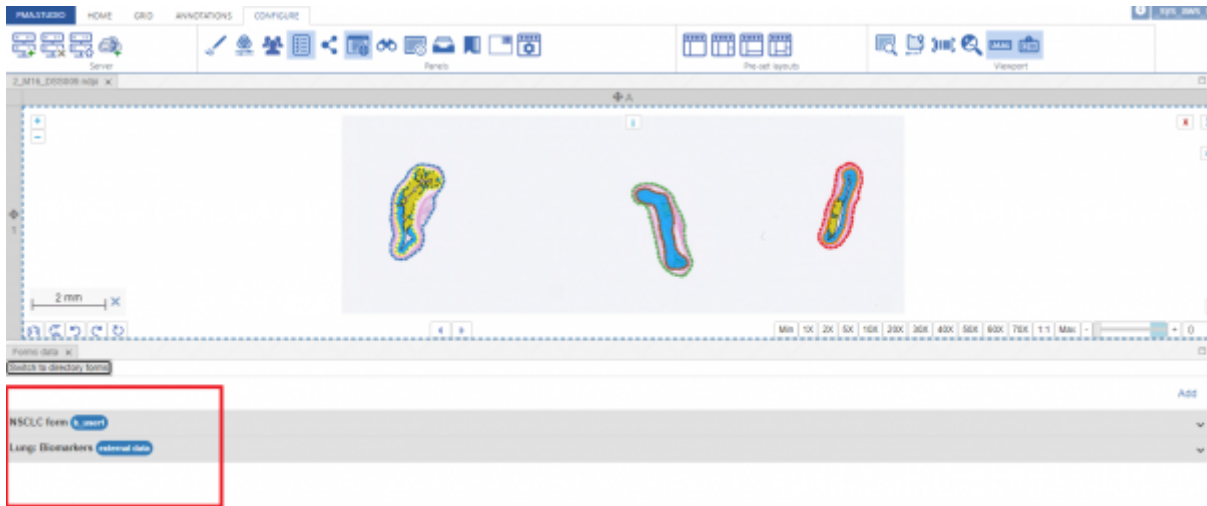

User can also click on the “edit” link on the right hand corner which will open the selected form in a tab where user can edit it.

Forms data	
Switch to directory forms	
Add	
NSCLC form <span>Learn</span>	
Edit	
Comments	test
TC %	1
TC cat	TC1
IC %	1
IC cat	IC2
Lung: Biomarkers <span>Get test data</span>	

Forms data × NSCLC form ×

Switch to directory forms

Server: PMA.core 3 dev

Slide: \_sys\_aws\_s3/annotations/ndpi-test/2\_M16\_DSS009.ndpi

### NSCLC form

**Comments**

**TC %**

**TC cat**

**IC %**

**IC cat**

From:  
<https://docs.pathomation.com/pimsls/3.1.0/> - PIMS LS 3.1.0

Permanent link:  
[https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=panel\\_forms&rev=1733480999](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=panel_forms&rev=1733480999)

Last update: **2024/12/06 13:29**

