

Slide information panel

The Info panel displays available slide information for currently selected slide.

Info	×	□	×
File name	_sys_ref/brightfield/Grundium/hp1 (1).svs		
Last Modified	Mar 01 2021 18:11		
Width	131,072 pixels / 32.840 mm		
Height	73,728 pixels / 18.473 mm		
Max Magnification	40X		
Micrometres Per Pixel X	0.250552		
Micrometres Per Pixel Y	0.250552		
Physical Size	328.53 MB		
Tile Size	512 x 512 pixels		
Barcode			
Barcode text			
Overview			
Macro	N/A		
Layers	1		
Channels	1		

Currently selected slide is highlighted with blue dashed border.

Columns are pretty self-descriptive. If information could not be determined, the field is left blank.

The barcode, macro and overview thumbnails are clickable. Clicking on the images will open then in a new browser tab.

The below screenshot shows the interface when barcode, macro, and overview thumbnails are all available. The barcode image is pulled up at a higher resolution in a separate browser window.

The barcode from the image is interpreted automatically and shown as a separate text-string in the info panel as well.

Info X	
File name	_sys_ref/projects/P0431/04310688C0010S.scn
Last Modified	Feb 13 2017 13:37
Width	36,832 pixels / 18.416 mm
Height	38,432 pixels / 19.216 mm
Max Magnification	20X
Micrometres Per Pixel X	0.5
Micrometres Per Pixel Y	0.5
Physical Size	291.81 MB
Tile Size	512 x 512 pixels
Barcode	
Barcode text	04050629C
Overview	
Macro	
Layers	1
Channels	1

barcode (404x350)

04050629C

08B19221-11



Exp: 09 Sep 2009

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=panel_info

Last update:

2023/03/03 13:11

