


# Shared Links Panel

PIMS LS allows you to [easily generate links to share slides with your collaborators](#).

To see and manage your shared links, you can open the Shared Links panel.

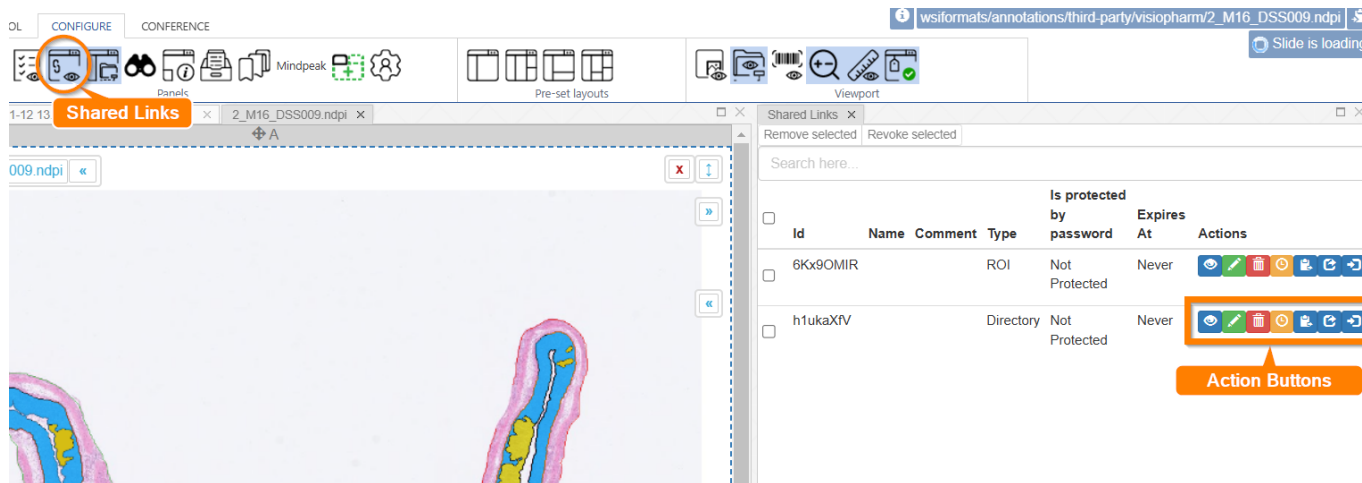

In this panel you will see a list of all currently active shared links, and actions for each

## Action Buttons

From left to right:

1. View slide - loads slide into viewport
2. Edit - opens a menu for editing the name, expiry date, password... etc of the slide
3. Delete - deletes the share
4. Suspend - leaves the share for you to edit, but revokes access / sets it to expired.
5. Copy to clipboard - copies link to your clipboard for you to paste elsewhere
6. Get Link - Shows the link and QR code of the link
7. Send to Tray - sends the slide in the link to your tray.

From:  
<https://docs.pathomation.com/pimsls/3.1.0/> - PIMS LS 3.1.0

Permanent link:  
[https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=panel\\_links&rev=1733479523](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=panel_links&rev=1733479523)

Last update: 2024/12/06 13:05

