

## Whats new?

### PIMS LS 2.2.0

#### Visual & Workflow Overhaul

##### Change

Much has been changed for this new incarnation of PIMSLS.

- The icon library has been completely revised
- The ribbon has been reorganised according to pathologist feedback to populate the home tab with most used features, and organize tabs in the order they are usually needed
- The annotation panel has been revised to get rid of dead space, and you can now proceed through annotations with the previous/next buttons
- The colour adjustment panel has been streamlined to make adjustments faster and easier
- The loading of annotations is made clearer to the user on annotation-heavy slides
- The software has been updated to reflect it's new name PIMSLS

#### Conference

##### Feature

The natural successor to the multi-head microscope is now available in the Conferencing tab.

[Navigate, annotate, and collaborate together on the same slide with just a couple of clicks.](#)

#### Annotation Text Editing

##### Change

Text annotating is now easier and you can dismiss the window if you just want to keep the placeholder text

#### Helpdesk links

##### Change

We've updated the links throughout the software to take you through to our helpdesk

#### Snapshots

##### Bugfix

Exporting snapshots to TIFF now exports in TIFF rather than PNG

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

[https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=release\\_notes&rev=1728655219](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=release_notes&rev=1728655219)

Last update: **2024/10/11 17:00**

