

Whats new?

PIMS LS 2.2.0

Visual & Workflow Overhaul

Change

Much has been changed for this new incarnation of PIMSLS.

- The icon library has been completely revised
- The ribbon has been reorganised according to pathologist feedback to populate the home tab with most used features, and organize tabs in the order they are usually needed
- The annotation panel has been revised to get rid of dead space, and you can now proceed through annotations with the previous/next buttons
- The colour adjustment panel has been streamlined to make adjustments faster and easier
- The loading of annotations is made clearer to the user on annotation-heavy slides
- The software has been updated to reflect it's new name PIMSLS

Conference

Feature

The natural successor to the multi-head microscope is now available in the Conferencing tab.

[Navigate, annotate, and collaborate together on the same slide with just a couple of clicks.](#)

Annotation Text Editing

Change

Text annotating is now easier and you can dismiss the window if you just want to keep the placeholder text

Helpdesk links

Change

We've updated the links throughout the software to take you through to our helpdesk

Snapshots

Bugfix

Exporting snapshots to TIFF now exports in TIFF rather than PNG

Are you missing some of these features in your version? Schedule an update today by contacting info@pathomation.com

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=release_notes&rev=1729496783

Last update: **2024/10/21 10:46**

