

## Whats new?

### PIMS LS 3.0.0

#### Complete overhaul of the ribbon

There is a complete overhaul of the ribbon in this release

- Streamlined dropdown menus
- Less clutter
- Quick access for pathologist's most-used features

#### AI support

\* Order, monitor and view results from third party AI algorithms, all from within PIMS LS \* Contact [info@pathomation.com](mailto:info@pathomation.com) or your account manager to set up integration with your contracted AI vendor(s) \* Whole slide or annotation-based analyses supported \* Usage restrictions e.g. 100 uses per user supported

#### Improvements to existing features

\* Grid snapshots are now customizable - you can place your scalebar where you like in your multi-image snapshots \* Max Magnification now bring you to the Maximum Optical magnification, not the max digitally enhanced zoom level \* Filenames are turned off on shares by default \* Autosaving annotations is now turned on by default for new users \* ROI / Annotation sharing now focuses on the annotation area that was shared

#### Bug fixes

\* Custom shares turn on/off display elements properly now \* Sliders for annotation line and fill opacity now adjust properly \* Repeated 'would you like to disable images layer transparency' messages have been fixed when adjusting annotation opacity \* A bug that sometimes caused the contextual menu for finishing/continuing Compound Freehand to disappear before you could click it has been fixed \* Slide popout now works as expected

From:  
<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:  
[https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=release\\_notes&rev=1746892955](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=release_notes&rev=1746892955)

Last update: **2025/05/10 19:02**

