

Whats new?

PIMS LS 3.0.0

Complete overhaul of the ribbon

There is a complete overhaul of the ribbon in this release

- Streamlined dropdown menus
- Less clutter
- Quick access for pathologist's most-used features

AI support

- Order, monitor and view results from third party AI algorithms, all from within PIMS LS
- Contact info@pathomation.com or your account manager to set up integration with your contracted AI vendor(s)
- Whole slide or annotation-based analyses supported
- Usage restrictions e.g. 100 uses per user supported

Improvements to existing features

- Grid snapshots are now customizable - you can place your scalebar where you like in your multi-image snapshots
- Max Magnification now bring you to the Maximum Optical magnification, not the max digitally enhanced zoom level
- Filenames are turned off on shares by default
- Autosaving annotations is now turned on by default for new users
- ROI / Annotation sharing now focuses on the annotation area that was shared

Bug fixes

- Custom shares turn on/off display elements properly now
- Sliders for annotation line and fill opacity now adjust properly
- Repeated 'would you like to disable images layer transparency' messages have been fixed when adjusting annotation opacity
- A bug that sometimes caused the contextual menu for finishing/continuing Compound Freehand to disappear before you could click it has been fixed
- Slide popout now works as expected

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=release_notes&rev=1746893040

Last update: **2025/05/10 19:04**

