

Ribbon > Annotations tab > Polygon shapes > Compound freehand

Button identification




Associated command

CompoundFreehand

Function description

Drawing polygon is the typical run off the mill polygon annotation tool you find in many graphics tools like Photoshop, Paint.Net etcetera. The tool, as implemented, assumes there's an automatic connection between your first and last drawn point. You finish drawing your shape by double-clicking.

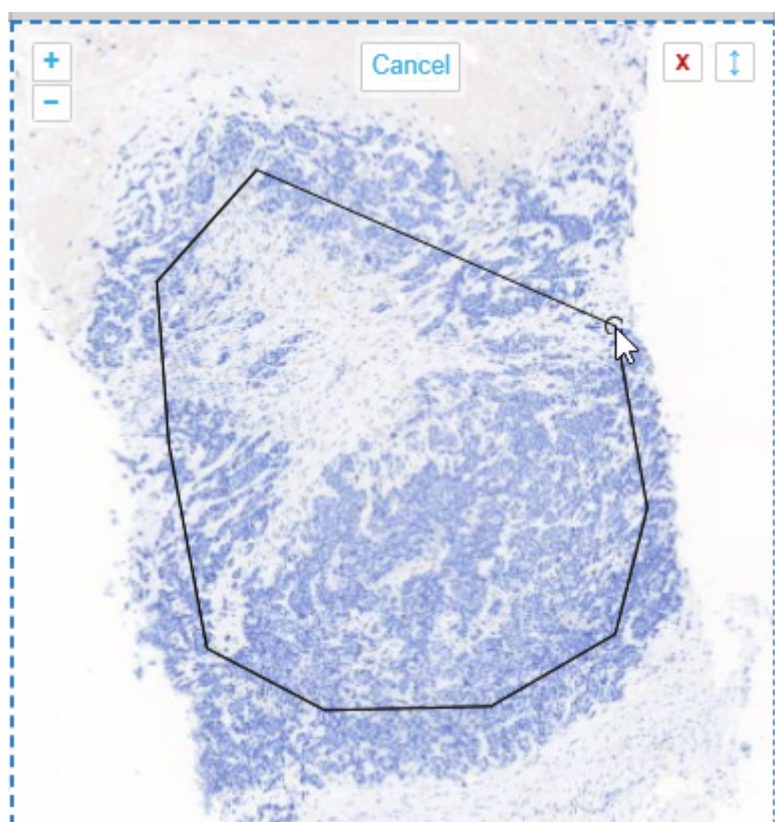

The [annotation browser](#) **doesn't** indicate of how many points the polygon shape is comprised.

Related functionality

[Ribbon > Annotations tab](#)

From:
<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:
https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=ribbon_annotations_compound_freehand

Last update: **2023/02/28 13:29**

