

## Available ribbon commands

See also the [tab inventory page](#).

Ribbon section	Command name	Parameters
File menu	custom-panel-algo-ichi	props=
	custom-panel-algo-ni	props=
	custom-panel-ai-prov-mindpeak	props=
	custom-panel-ai-prov-spintellx	props=
	file	
	about-us	
	show-eula	
	manual	
	sign-out	
Ribbon section	Command name	Parameters
Home tab	simple-layout	
	detailed-layout	
	detailed-layout	
	viewport-toggle-overview	
	viewport-toggle-slide-path	
	viewport-toggle-barcode	
	viewport-toggle-rotation-flip	
	add-tab	
	reset-layout	
	viewport-toggle-zoom	
	viewport-toggle-scalebar	
	viewport-toggle-label	
	panel-toggle-color	
	panel-toggle-search	
	open-folder-in-grid	
	prev-directory	
	next-directory	
	prev-image	
	next-image	
	add-folder-bookmark	
	reset-grid	
	clear-cell	
	open-in-tab	
	popout-info	
	snapshot	
	quick-snapshot	
	toggle-toolbar	
increase-zoom		
decrease-zoom		
custom-zoom-1x	pma-magnification=	
custom-zoom-2x	pma-magnification=	

<b>Ribbon section</b>	<b>Command name</b>	<b>Parameters</b>
	<a href="#">custom-zoom-5x</a>	pma-magnification=
	<a href="#">custom-zoom-10x</a>	pma-magnification=
	<a href="#">custom-zoom-20x</a>	pma-magnification=
	<a href="#">custom-zoom-40x</a>	pma-magnification=
	<a href="#">panel-toggle-color</a>	
	<a href="#">send-slide-to-tray</a>	
	<a href="#">send-to-tray</a>	
	<a href="#">panel-toggle-tray</a>	
	<a href="#">share-slide-tray</a>	
	<a href="#">quick-share-folder</a>	
	<a href="#">quick-share-slide</a>	
	<a href="#">quick-share-slide-roi</a>	
	<a href="#">quick-share-grid</a>	
	<a href="#">quick-share-grid-rois</a>	
	<a href="#">quick-share-thumbnail</a>	
	<a href="#">share-slide</a>	
<b>Ribbon section</b>	<b>Command name</b>	<b>Parameters</b>
<a href="#">Grid tab</a>	<a href="#">toggle-1x1-grid</a>	
	<a href="#">toggle-2x2-grid</a>	
	<a href="#">toggle-3x3-grid</a>	
	<a href="#">sync-views-toggle</a>	
	<a href="#">reset-grid</a>	
	<a href="#">clear-cell</a>	
	<a href="#">open-in-tab</a>	
	<a href="#">viewport-add-row</a>	
	<a href="#">viewport-remove-row</a>	
	<a href="#">viewport-remove-row</a>	
	<a href="#">viewport-remove-column</a>	
	<a href="#">flip-rotate-menu</a>	
	<a href="#">reset-flip</a>	
	<a href="#">flip-horizontally</a>	
	<a href="#">flip-vertically</a>	
	<a href="#">reset-rotation</a>	
	<a href="#">rotate-clockwise-90</a>	
	<a href="#">rotate-counter-clockwise-90</a>	
	<a href="#">rotate-180</a>	
	<a href="#">zoom-in</a>	
	<a href="#">panel-toggle-annotations</a>	
	<a href="#">save-annotations</a>	
	<a href="#">toggle-annotations</a>	
	<a href="#">toggle-annotation-text</a>	
	<a href="#">toggle-fingerprint</a>	
<b>Ribbon section</b>	<b>Command name</b>	<b>Parameters</b>
<a href="#">Annotations tab</a>	<a href="#">Cursor</a>	
	<a href="#">delete-annotation</a>	

<b>Ribbon section</b>	<b>Command name</b>	<b>Parameters</b>
	<a href="#">delete-all-annotations</a>	
	<a href="#">edit-annotation-text</a>	
	<a href="#">merge-selected-annotations</a>	
	<a href="#">union-selected-annotations</a>	
	<a href="#">modify-selected-annotation</a>	
	<a href="#">Point</a>	
	<a href="#">Circle</a>	
	<a href="#">Line</a>	
	<a href="#">Arrow</a>	
	<a href="#">Ellipse</a>	
	<a href="#">Rectangle</a>	
	<a href="#">Landmark1</a>	
	<a href="#">MultiPoint</a>	
	<a href="#">Polygon</a>	
	<a href="#">ClosedFreehand</a>	
	<a href="#">CompoundFreehand</a>	
	<a href="#">Freehand</a>	
	<a href="#">Wand</a>	
	<a href="#">Brush</a>	
	<a href="#">export-to-wkt</a>	
	<a href="#">export-to-mld</a>	
	<a href="#">export-to-annotations</a>	
	<a href="#">export-to-aperio</a>	
	<a href="#">export-to-csv</a>	
	<a href="#">import-to-wkt</a>	
	<a href="#">import-to-mld</a>	
	<a href="#">import-to-halo</a>	
	<a href="#">import-to-csv</a>	
<b>Ribbon section</b>	<b>Command name</b>	<b>Parameters</b>
<a href="#">Pre-set tab</a>	<a href="#">preset-Circle0237mm2</a>	pma-shape, pma-size-a, pma-size-b
	<a href="#">preset-Circle1mm</a>	pma-shape, pma-size-a, pma-size-b
	<a href="#">preset-Circle2mm</a>	pma-shape, pma-size-a, pma-size-b
	<a href="#">preset-Circle1cm</a>	pma-shape, pma-size-a, pma-size-b
	<a href="#">preset-Square5mm</a>	pma-shape, pma-size-a, pma-size-b
	<a href="#">preset-Square1cm</a>	pma-shape, pma-size-a, pma-size-b
	<a href="#">preset-Tumor</a>	pma-classification, pma-shape, pma-color
	<a href="#">preset-Macro</a>	pma-classification, pma-shape, pma-color
<b>Ribbon section</b>	<b>Command name</b>	<b>Parameters</b>
<a href="#">Conferencing tab</a>	<a href="#">create-conference</a>	
	<a href="#">join-conference</a>	
	<a href="#">start-conference</a>	
	<a href="#">pause-conference</a>	
	<a href="#">stop-conference</a>	
	<a href="#">toggle-everyone-control-conference</a>	
	<a href="#">broadcast-conference</a>	

Ribbon section	Command name	Parameters
	<a href="#">panel-toggle-conference</a>	
	<a href="#">panel-toggle-forms</a>	
	<a href="#">panel-toggle-links</a>	
	<a href="#">panel-toggle-navigation</a>	
	<a href="#">panel-toggle-bookmarks</a>	

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:

[https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=ribbon\\_commands&rev=1680715201](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=ribbon_commands&rev=1680715201)

Last update: **2023/04/05 20:20**

