


# Ribbon Config

PMA.studio allows to configure Ribbon layout through the admin side. Once you are logged in as an admin and click on “Ribbon Layout” from the left menu, you are taking to Ribbon layout configuration page. Through this page you can specify layout of PMA.studio UI by adding Menu or tabs in xml format that the front-end application parses accordingly. Each ribbon section consists of set of command tags that are parsed into set of controls on the UI.

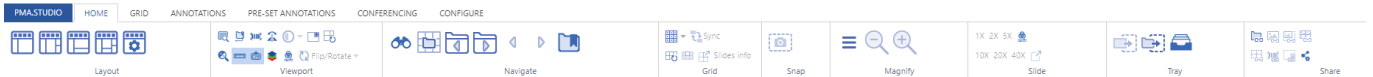

From: <https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link: [https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=ribbon\\_config&rev=1652037500](https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=ribbon_config&rev=1652037500)

Last update: **2022/05/08 22:18**

