



Grid: Size Tool Group

The Size dropdown tool contains icons for changing how many cells are visible in your viewport

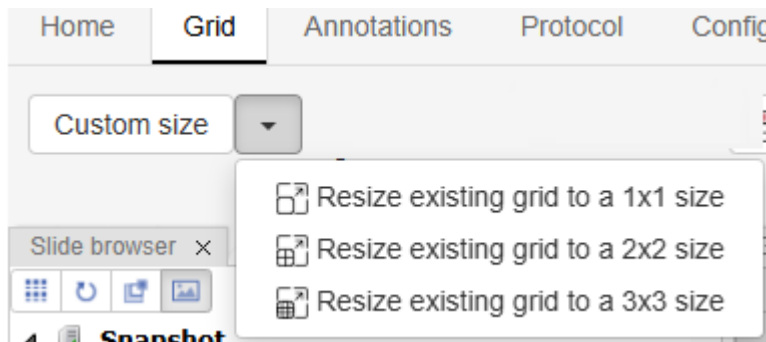


User can make a 1x1, 2x2, and 3x3 viewport grid, with 1, 4 and 9 images - respectively. The 2x2 option is shown below:

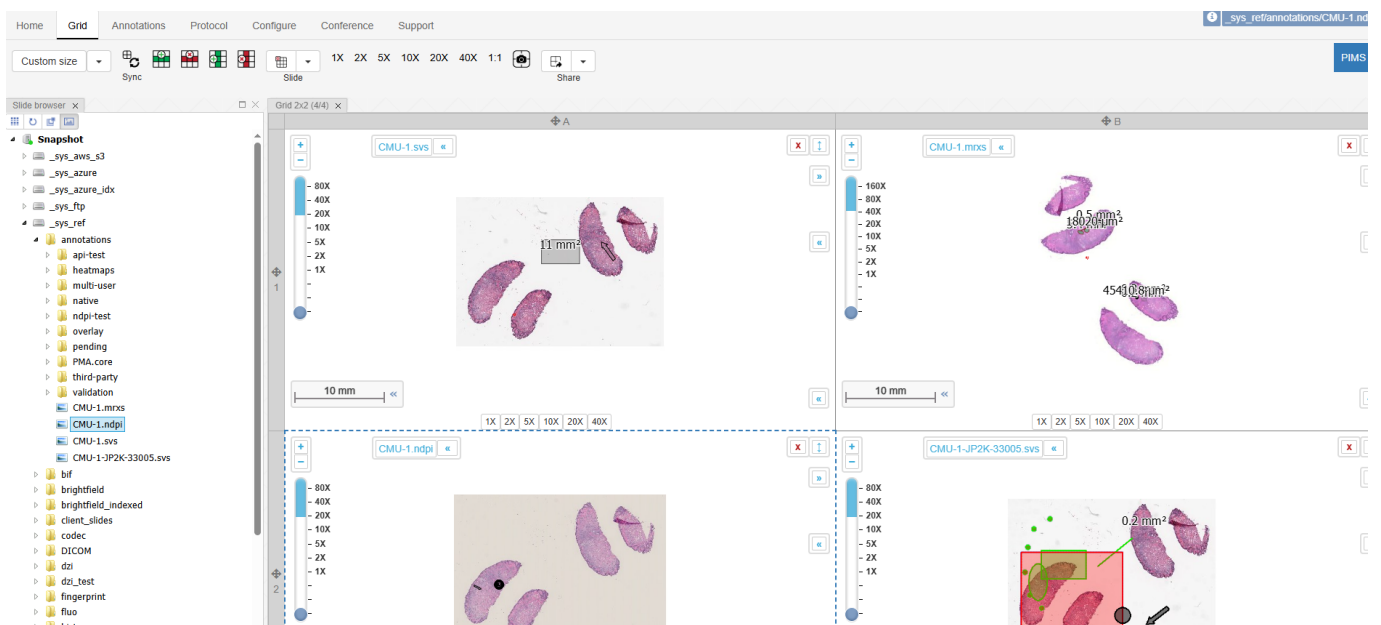

When you click one of the options, the software will ask if you want to keep the current images or replace them.

Bottom row

The custom size option allows you select how many cells tall and wide you want your viewport to be.

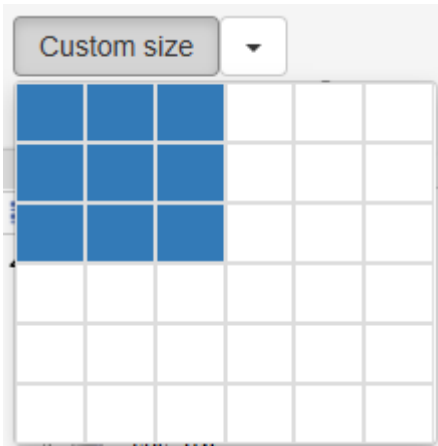

From:
<https://docs.pathomation.com/pimsls/3.1.0/> - **PIMS LS 3.1.0**

Permanent link:
<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=size&rev=1746904482>

Last update: **2025/05/10 22:14**

