


## Grid: Slide Tool Group

The Slide toolgroup contains tools for turning changing brightness/contrast of your current image, rotating and magnifying the slide.

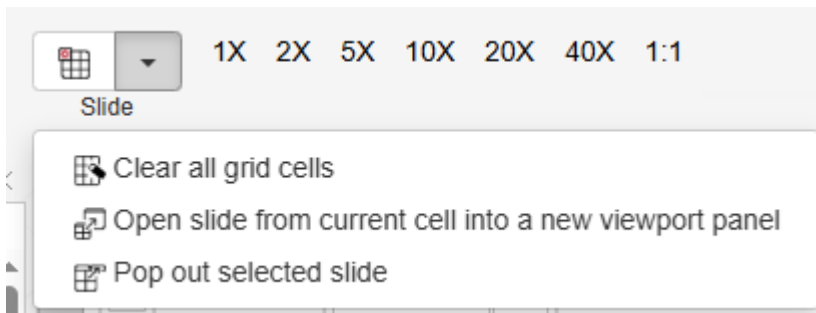

### Top Row

1. Brightness/contrast adjuster
2. Flip/Rotate dropdown - opens a dropdown menu with all rotation commands
3. Pop out selected/current slide
4. Send current slide to Tray [read more about trays here](#)

### Bottom Row

Here are commands to control the magnification level

1. Minimum magnification
2. Minus one magnification level
3. Buttons for each objective magnification level from 1x to 40x
4. Plus one magnification level
5. Maximum magnification

From:

<https://docs.pathomation.com/pimsls/3.1.0/> - PIMS LS 3.1.0

Permanent link:

<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=slide&rev=1746905020>

Last update: **2025/05/10 22:23**

