


# Annotations: Style Workgroup

The Annotation style section contains icons for configuring annotation colour and transparency.

All annotation styles are set by clicking the icon and choosing a colour, number, or percentage from the dropdown menu.

## Annotation colour

Clicking the annotation colour lets you choose the outline colour of your annotation

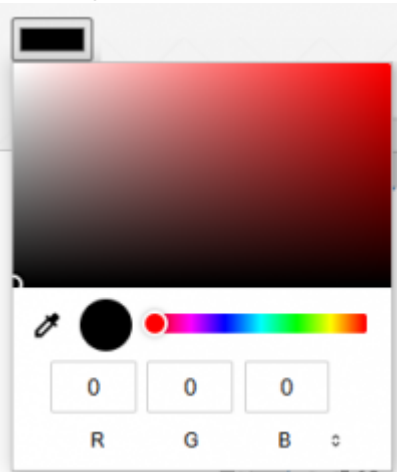


## Annotation thickness

Clicking the annotation thickness lets you choose the thickness of the line of your annotation




You can choose a number from 1-10 in the dropdown menu:

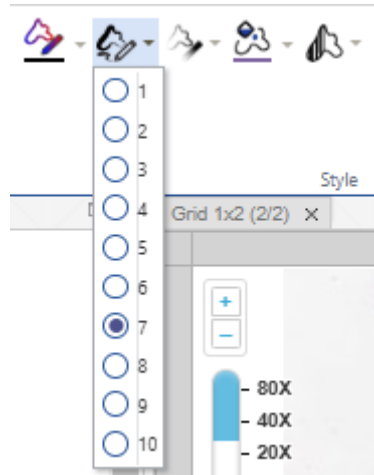


### Annotation opacity

Clicking the annotation opacity lets you choose how opaque/transparent your annotation outline is

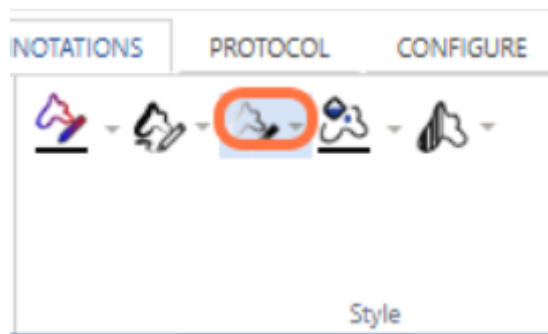


### Fill colour

Clicking the fill colour lets you choose the fill colour of your annotation (if its a closed shape)

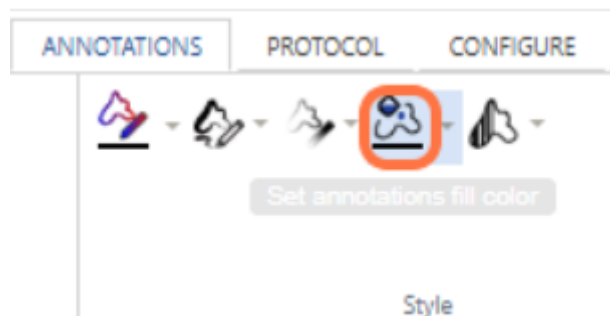


You can select a colour from the dropdown menu:

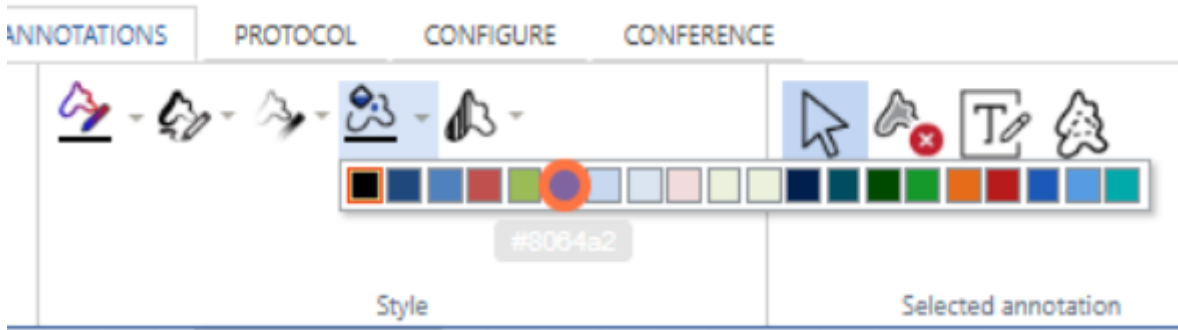

### Fill opacity

Clicking the fill opacity lets you choose the fill opacity/transparency for your closed shape annotations e.g. square, closed freehand

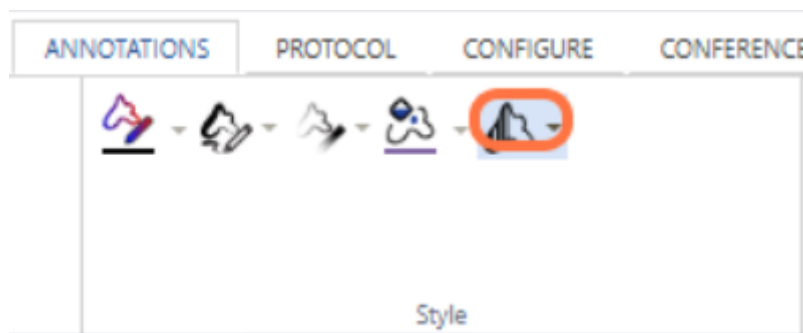

From:  
<https://docs.pathomation.com/pimsls/3.1.0/> - PIMS LS 3.1.0

Permanent link:  
<https://docs.pathomation.com/pimsls/3.1.0/doku.php?id=style&rev=1746908424>

Last update: 2025/05/10 23:20

